

Database

INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT

Serial No.: 80-004206 Instrument Location: Ward County Jail ✓

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
X Other (specify) New

Check When Done:

- X A. Surge Protector Installed/Properly Grounded.
- X B. Telephone Line Connected to Intoxilyzer® KB.
- X C. Breath Tube Heated.
- X D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- X E. Ethanol Breath Standard: X Connected X Installed.
(Include Form 105-G if New Cylinder Installed.)
- X F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- X G. Print Test (Level 1, Function P). (Attach Test Record.)
- X H. ACA Test (Level 1, Function C). (Attach Test Record.)
- X I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: N/A

10/08/09
Date
08 Oct 09
Date

Patricia Kasha
Field Inspector's Signature
Deb Kasha
Reviewed By

Mike McCloud will be bringing this intox in to swap for another 8000. OK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = WARD 8164.12.00 06/09
10/02/2009 20:57

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659357
Standard Cyl #? 11
Standard Expiration? 09/03/2011
Oper No? 049509

Flow Cal. Date: 06/10/2009
Slope 690
Intercept -645883

IR Calibration Date: 06/10/2009

	3um	9um
0th Coef(*100):	-16171	-25317
1st Coef(*100):	259064	128821
2nd Coef(*100):	2464	1154
H2O adj(mg/l*10k):	521	622

***** Printer Test End *****

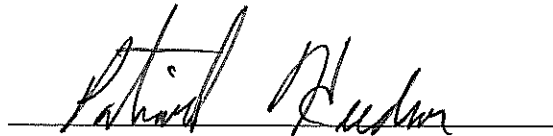

Operator Signature
PATRICK C. HUDSON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = WARD 8164.12.00 06/09
10/02/2009 21:03

Test	AC	Time
01 Diagnostic	OK	21:05
02 Room Air	RFI*	21:05
03 Room Air	0.000	21:06

*Invalid Test
Inhibited - RFI

Sub Name = TEST, SUBJECT NA
Sub DOB = 02/02/1970
Sub Sex = Male Weight = NA
Test = OTH Cit = 1234567
Dr. Lic. = ND/2345678
Lot No = 659357
Cyl No = 11
Expiration Date = 09/03/2011
County = 51 Oper No. = 049509

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

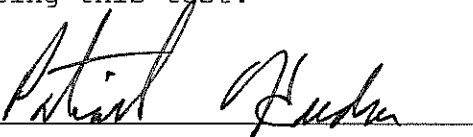

Operator Signature
PATRICK C. HUDSON

Remarks:

Form 106-I8000

80-004206
P. 3 of 3 pdr