



# ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59282 (5-2011)

Chemical Test Operator Name (Print) <b>MATTHEW W. ANDERSON</b>		
Location <b>Richland County Jail</b>	Intoxilyzer® Serial Number <b>80-005947</b>	
Gas Lot Number <b>116512080A2</b>	Gas Cylinder Number <b>58</b>	Gas Expiration Date <b>8/01/14</b>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
A. 0. 081 AC  
B. 0. 081 AC  
C. 0. 082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Matthew W. Anderson</i>	Date <del>10/02/14</del> 10/02/13 <sup>DK</sup>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date 11 Oct 2013


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005947  
Location = WAHP      8164.13.00 06/09  
10/02/2013      13:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:55
02 Std. Gas	0.081	13:55
03 Room Air	0.000	13:56
04 Std. Gas	0.081	13:56
05 Room Air	0.000	13:57
06 Std. Gas	0.082	13:57
07 Room Air	0.000	13:58

Lot No = 16512080A2  
Cyl No = 58  
Exp Date = 08/01/2014  
County = 39      Oper No. = 020401

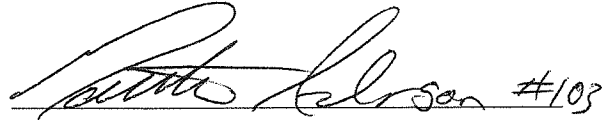

Operator Signature  
MATTHEW ANDERSON

Remarks:

*1st ACA, Good test*

Form 106-I8000