

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004198

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No.      Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No.      Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No.      Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 659358 Cyl No. 54 Exp. Date 03 Sept 2011

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>

Average 0.080

L. Remarks/Maintenance record: Annual inspection, Software 8164.13 upgrade & calibrations (attached) <sup>DK</sup>

10 Dec 2009 Deb Kashur  
Date Field Inspector's Signature

15 Dec 2009 Mary Pearson  
Date Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-004198  
Location = TOXL                      8164.13.00 06/09  
12/09/2009                              14:08

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*( )\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*( )\_+?

Current Instrument Setup

Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?            No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?            0.080  
Standard Lot #?            659358  
Standard Cyl #?            54  
Standard Expiration?      09/03/2011  
Oper No?                    777777

Flow Cal. Date:            06/04/2009  
Slope                       671  
Intercept                  -536290

IR Calibration Date:       12/09/2009  
                             3um                            9um  
-----  
0th Coef(\*100):            -31070                      -34849  
1st Coef(\*100):            268664                      135239  
2nd Coef(\*100):            -643                            440  
H2O adj(mg/l\*10k):       570                           615

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

*Print Test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004198  
Location = TOXL      8164.13.00 06/09  
12/09/2009      14:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:40
02 Std. Sol.	0.020	14:40
03 Room Air	0.000	14:41
04 Std. Sol.	0.020	14:42
05 Room Air	0.000	14:42
06 Std. Sol.	0.019	14:43
07 Room Air	0.000	14:43

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

Low AC 0.019 AC

Form 106-18000

80-004198  
P. 3 of 9 PM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004198  
Location = TOXL                 8164.13.00 06/09  
12/09/2009                        14:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:51
02 Std. Sol.	0.286	14:52
03 Room Air	0.000	14:52
04 Std. Sol.	0.287	14:53
05 Room Air	0.000	14:53
06 Std. Sol.	0.287	14:54
07 Room Air	0.000	14:55


08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

*Linearity test*

Form 106-18000

*80-004198  
P. 4/9/09*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004198  
Location = TOXL                              8164.13.00 06/09  
12/09/2009                                      16:05

Test	AC	Time
01 Room Air	0.000	16:05
02 *Subject Test	INT*	16:06
03 Room Air	0.000	16:06

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI                                      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 54  
Expiration Date = 09/03/2011  
County = 08                                      Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

*Interferent test*

Form 106-1B000

80-004198  
P. 5 of 9 Jan

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004198  
Location = TOXL                              8164.13.00 06/09  
12/10/2009                                      09:13

Test	AC	Time
01 Diagnostic	OK	09:14
02 Room Air	0.000	09:14
03 *Subject Test	RFI*	09:15
04 Room Air	0.000	09:15

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI                                      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 54  
Expiration Date = 09/03/2011  
County = 08                                      Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

RFI test

Form 106-18000

80-004198  
P. 6 of 9 PL

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004198  
Location = TOXL                8164.13.00 06/09  
12/09/2009                        15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.080	15:02
03 Room Air	0.000	15:03
04 Std. Gas	0.079	15:03
05 Room Air	0.000	15:04
06 Std. Gas	0.080	15:04
07 Room Air	0.000	15:04

Lot No = 659358  
Cyl No = 54  
Exp Date = 09/03/2011  
County = 08                            Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-18000

*80-004198  
P. Def 9/10/09*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004198  
Location = TOXL                8164.13.00 06/09  
12/09/2009                        15:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:05
02 Std. Gas	0.080	15:06
03 Room Air	0.000	15:06
04 Std. Gas	0.080	15:07
05 Room Air	0.000	15:07
06 Std. Gas	0.080	15:07
07 Room Air	0.000	15:08

Lot No = 659358  
Cyl No = 54  
Exp Date = 09/03/2011  
County = 08                      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-18000

80-004198  
P. 849 KM



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004198  
Location = TOXL                8164.13.00 06/09  
12/09/2009                        15:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:50
02 Std. Gas	0.081	15:50
03 Room Air	0.000	15:50
04 Std. Gas	0.080	15:51
05 Room Air	0.000	15:51
06 Std. Gas	0.080	15:52
07 Room Air	0.000	15:52

Lot No = 659358  
Cyl No = 54  
Exp Date = 09/03/2011  
County = 08                      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-18000

80-004198  
P. 9 of 9 JAK