



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Tracy L. Meidinger</i>		
Location <i>UND-PD</i>	Intoxilyzer® Serial Number <i>80-004942</i>	
Gas Lot Number <i>14214080A6</i>	Gas Cylinder Number <i>58</i>	Gas Expiration Date <i>07-05-16</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.082* AC
  - B. *0.082* AC
  - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Capt. Tracy L. Meidinger</i>	Date <i>10/16/2014</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>20 Oct 2014</i>

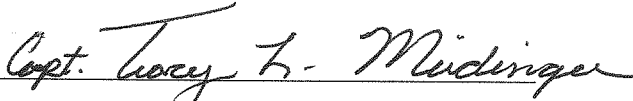
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = UNDP      8164.13.00 06/09  
10/16/2014      16:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:25
02 Std. Gas	0.082	16:25
03 Room Air	0.000	16:26
04 Std. Gas	0.082	16:26
05 Room Air	0.000	16:27
06 Std. Gas	0.081	16:27
07 Room Air	0.000	16:27

Lot No = 14214080A6  
Cyl No = 58  
Exp Date = 07/05/2016  
County = 18      Oper No. = 119409

  
Operator Signature  
TRACY MEIDINGER

Remarks: