



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (1-2010)

Serial Number: 80-003071 Instrument Location: TIOGA

Reason for Install/Repair:

Install After Lab Repair/Inspection

Other (Specify) INSTALL ANOTHER UNIT

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: \_\_\_\_\_

[Signature] \_\_\_\_\_ Date 10/24/10

Field Inspector Signature

Deb Kashur \_\_\_\_\_ Date 27 Oct 2010

Reviewed By

80-003071  
p. 102 of 105  
JAM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TIOG      8164.13.00 06/09  
10/24/2010      01:14

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      15410080A1  
Standard Cyl #?      25  
Standard Expiration?      07/01/2012  
Oper No?      020632

Flow Cal. Date:      07/09/2010  
Slope      806  
Intercept      -994737

IR Calibration Date:      07/09/2010

	3um	9um
0th Coef(*100):	-16015	-11994
1st Coef(*100):	278129	138369
2nd Coef(*100):	1927	1066
H2O adj(mg/l*10k):	293	137

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
SHANNON P. DUISEN

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TIOG      8164.13.00 06/09  
10/24/2010      01:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:15
02 Std. Gas	0.078	01:16
03 Room Air	0.000	01:16
04 Std. Gas	0.078	01:17
05 Room Air	0.000	01:17
06 Std. Gas	0.078	01:17
07 Room Air	0.000	01:18

Lot No = 15410080A1  
Cyl No = 25  
Exp Date = 07/01/2012  
County = 53      Oper No. = 020632



Operator Signature  
SHANNON P. DUISEN

Remarks:

2<sup>nd</sup> ACA

Form 106-I8000

80-003071  
P. 104 of 105  
SPM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TIOG      8164.13.00 06/09  
10/24/2010      01:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	01:30
02 Room Air	RFI*	01:30

\*Invalid Test  
Inhibited - RFI

Lot No = 15410080A1  
Cyl No = 25  
Exp Date = 07/01/2012  
County = 53      Oper No. = 020632



Operator Signature  
SHANNON P. DUISEN

Remarks:

RFI

Form 106-I8000

80-003071  
P. 105 of 105  
JRM