

✓ DATABASES
10-29-07
CML

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-004198 Instrument Location: Richland County Jail


Reason for Install/Repair: Install After Lab Repair/Inspection
 Other (specify) _____

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® KB.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Ethanol Breath Standard: Connected _____ Installed.
(Include Form 105-G if New Cylinder Installed.)
- F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- G. Print Test (Level 1, Function P). (Attach Test Record.)
- H. ACA Test (Level 1, Function C). (Attach Test Record.)
- I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: Installed new

intoxilyzer 80-004198

10-26-2009
Date
29 Oct 2009
Date


Field Inspector's Signature
Deb Kashur
Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = WAHP 8164.12.00 06/09
10/26/2009 23:10

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.077
Standard Lot #? 659358
Standard Cyl #? 52
Standard Expiration? 09/03/2011
Oper No? 020401

Flow Cal. Date: 06/04/2009
Slope 671
Intercept -536290

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100): -19450 -26720
1st Coef(*100): 261998 131587
2nd Coef(*100): 2424 1229
H2O adj(mg/l*10k): 588 585

***** Printer Test End *****

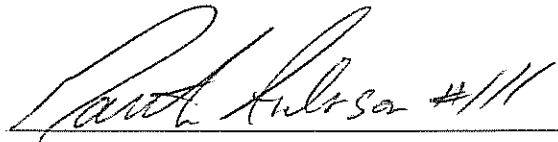

Operator Signature
MATTHEW ANDERSON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = WAHP 8164.12.00 06/09
10/26/2009 23:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	23:22
02 Std. Gas	0.082	23:23
03 Room Air	0.000	23:23
04 Std. Gas	0.082	23:24
05 Room Air	0.000	23:24
06 Std. Gas	0.081	23:25
07 Room Air	0.000	23:25

Lot No = 659358
Cyl No = 52
Exp Date = 09/03/2011
County = 39 Oper No. = 020401

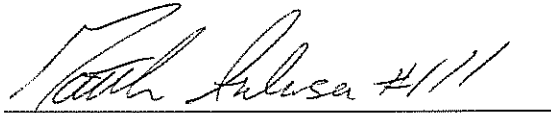

Operator Signature
MATTHEW W. ANDERSON

Remarks:

Form 106-I8000

P. 3 of 4
JFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004198
Location = WAHP 8164.12.00 06/09
10/26/2009 23:17

Test	AC	Time
01 Diagnostic	OK	23:19
02 Room Air	RFI*	23:19
03 Room Air	0.000	23:19

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NMN
Sub DOB = 03/20/1979
Sub Sex = Male Weight = 260
Test = DUI Cit = 234567
Dr. Lic. = ND/12345678
Lot No = 659358
Cyl No = 52
Expiration Date = 09/03/2011
County = 39 Oper No. = 020401

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

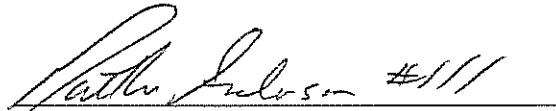

Operator Signature
MATTHEW W. ANDERSON

Remarks:

Form 106-I8000