



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number <i>80-003065</i>	Instrument Location <i>GFSO</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <i>Printer Change</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>S. Nelson</i>	Date <i>10/31/14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>10 Nov 14</i>

Install was not needed however completed to ensure everything is working properly which it is.

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Ede* Certified Date: *07 Nov. 16*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
10/31/2014 14:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:30
02 Std. Gas	0.082	14:30
03 Room Air	0.000	14:31
04 Std. Gas	0.081	14:31
05 Room Air	0.000	14:32
06 Std. Gas	0.081	14:32
07 Room Air	0.000	14:33

Lot No = 33913080A4
Cyl No = 47
Exp Date = 01/01/2016
County = 18 Oper No. = 039804



Operator Signature
LAVONNE NELSON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
10/31/2014 14:36

Test	AC	Time
01 Diagnostic	OK	14:37
02 Room Air	RFI*	14:38
03 Room Air	0.000	14:38

*Invalid Test
Inhibited - RFI

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

6-C

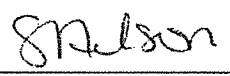

Operator Signature
LAVONNE NELSON

Remarks: