



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <i>80-004942</i>	Instrument Location <i>NcPO</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>G. Ash</i>	Date <i>11-4-11</i>
Reviewed By (Crime Laboratory Use Only) <i>G. Deb Keshur</i>	Date <i>08 NOV 2011</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = NAPO 8164.13.00 06/09
11/04/2011 10:16

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 29
Standard Expiration? 10/01/2013
Oper No? 120620

Flow Cal. Date: 01/20/2010
Slope 699
Intercept -690752

IR Calibration Date: 06/14/2011
 3um 9um

0th Coef(*100): -4912 -8076
1st Coef(*100): 251336 126802
2nd Coef(*100): 4293 1888
H2O adj(mg/l*10k): 593 646

***** Printer Test End *****



Operator Signature
GORDON LECLAIR

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = NAPO 8164.13.00 06/09
11/04/2011 11:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:53
02 Std. Gas	0.081	11:53
03 Room Air	0.000	11:54
04 Std. Gas	0.080	11:54
05 Room Air	0.000	11:54
06 Std. Gas	0.081	11:55
07 Room Air	0.000	11:55

Lot No = 23411080A1
Cyl No = 29
Exp Date = 10/01/2013
County = 24 Oper No. = 120620



Operator Signature
GORDON LECLAIR

Remarks: ACA

