

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-004191 Instrument Location: TRAIL CO S.O.

Reason for Install/Repair: _____ Install After Lab Repair/Inspection

X Other (specify) New Intoxilyzer 8000

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® KB.
- C. Breath-Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Ethanol Breath Standard: Connected Installed.
(Include Form 105-G if New Cylinder Installed.)
- F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- G. Print Test (Level 1, Function P). (Attach Test Record.)
- H. ACA Test (Level 1, Function C). (Attach Test Record.)
- I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: New

INTOXILYZER 8000

11-7-09
Date

17 NOV 09
Date

Ray W. Jelen
Field/Inspector's Signature

Deb Keshur
Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = HILL 8164.12.00 06/09
11/07/2009 03:01

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes

of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 45
Standard Expiration? 09/03/2011
Oper No? 058405

Flow Cal. Date: 06/03/2009
Slope 710
Intercept -670706

IR Calibration Date: 06/03/2009
3um 9um

0th Coef(*100): -5389 -24455
1st Coef(*100): 268636 131046
2nd Coef(*100): 2601 1309
H2O adj(mg/l*10k): 391 565

***** Printer Test End *****



Operator Signature
RAY G. WEBER

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = HILL 8164.12.00 06/09
11/07/2009 03:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	03:04
02 Std. Gas	0.079	03:04
03 Room Air	0.000	03:05
04 Std. Gas	0.081	03:05
05 Room Air	0.000	03:06
06 Std. Gas	0.081	03:06
07 Room Air	0.000	03:07

Lot No = 659358
Cyl No = 45
Exp Date = 09/03/2011
County = 49 Oper No. = 058405

Ray Weber

Operator Signature
RAY G. WEBER

Remarks: *ACA TEST*

Form 106-I8000

*80-004191
p. 3 of 4 AL*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004191
Location = HILL 8164.12.00 06/09
11/07/2009 03:09

Test	AC	Time
01 Diagnostic	OK	03:11
02 Room Air	RFI*	03:12
03 Room Air	0.000	03:12

*Invalid Test
Inhibited - RFI

Sub Name = RFI TEST, NA NA
Sub DOB = 02/02/2009
Sub Sex = Male Weight = NA
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 659358
Cyl No = 45
Expiration Date = 09/03/2011
County = 49 Oper No. = 058405

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
RAY G. WEBER

Remarks: *RFI TEST*

Form 106-18000

*80-004191
P. 4 of 4 FL*