



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (1-2010)

Serial Number: 80-004201 Instrument Location: GFAF

Reason for Install/Repair:

- Install After Lab Repair/Inspection
- Other (Specify) moved 9000 new building

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: N/A

Will. Pott

Field Inspector Signature

7 Nov 12

Date

Deb Kashur

Reviewed By

31 Jan 2013

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAP 8164.13.00 06/09
11/07/2012 18:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05911080A1
Standard Cyl #? 10
Standard Expiration? 02/01/2013
Oper No? 120011

Flow Cal. Date: 06/03/2009
Slope 696
Intercept -646527

IR Calibration Date: 04/18/2012
 3um 9um

0th Coef(*100): -12334 -9834
1st Coef(*100): 257748 124056
2nd Coef(*100): 7079 2216
H2O adj(mg/l*10k): 716 632

***** Printer Test End *****

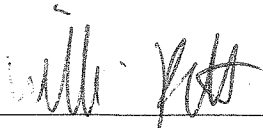

Operator Signature
WILLIAM R. PETTY

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAF 8164.13.00 06/09
11/07/2012 19:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:01
02 Std. Gas	0.077	19:01
03 Room Air	0.000	19:01
04 Std. Gas	0.077	19:02
05 Room Air	0.000	19:02
06 Std. Gas	0.078	19:03
07 Room Air	0.000	19:03

Lot No = 05911080A1
Cyl No = 10
Exp Date = 02/01/2013
County = 65 Oper No. = 120011

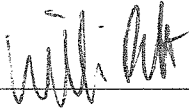

Operator Signature
WILLIAM R. PETTY

Remarks:

ACA Test Re-install

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004201
Location = GFAF 8164.13.00 06/09
11/07/2012 19:05

Test	AC	Time
01 Diagnostic	OK	19:07
02 Room Air	RFI*	19:07
03 Room Air	RFI*	19:07

*Invalid Test
Inhibited - RFI

Sub Name = TEST, INSTALL NA
Sub DOB = 10/31/2000
Sub Sex = Male Weight = 100
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 05911080A1
Cyl No = 10
Expiration Date = 02/01/2013
County = 65 Oper No. = 120011

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

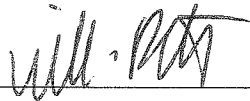

Operator Signature
WILLIAM R. PETTY

Remarks: