

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-003057

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No. — Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No. — Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 659358 Cyl No. 36 Exp. Date 03 Sept 2011

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>

Average 0.082

L. Remarks/Maintenance record: New software version 8164.13

09 NOV 09  
Date

Deb Kashur  
Field Inspector's Signature

1 Dec 09  
Date

Mary Pearson  
Reviewed by

80-003057  
P. 1 of 9  
YHL

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003057  
Location = TOXL                8164.13.00 06/09  
11/09/2009                      10:19

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            659358  
Standard Cyl #?            35  
Standard Expiration?       09/03/2011  
Oper No?                    777777

Flow Cal. Date:            04/07/2008  
Slope                      718  
Intercept                  -644065

IR Calibration Date:       06/05/2008  
                                  3um                            9um

-----  
0th Coef(\*100):            -22884                        -16302  
1st Coef(\*100):            265055                        140885  
2nd Coef(\*100):            151                            572  
H2O adj(mg/l\*10k):        508                            293

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      10:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:31
02 Std. Sol.	0.018	10:46
03 Room Air	0.000	10:47
04 Std. Sol.	0.018	10:47
05 Room Air	0.000	10:48
06 Std. Sol.	0.019	10:49
07 Room Air	0.000	10:49

08 Sim Temp = 34.0°C

DRS144 DK  
Simul Ser No = DR3409  
Std Sol No = 000  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:  
Low AC

Form 106-I8000

80-003057  
P. 3 of 9  
PK

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      11:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:17
02 Std. Sol.	0.287	11:18
03 Room Air	0.000	11:19
04 Std. Sol.	0.289	11:20
05 Room Air	0.000	11:20
06 Std. Sol.	0.289	11:21
07 Room Air	0.000	11:21

08 Sim Temp = 34.0°C

Simul Ser No = 000000  
Std Sol No = 000  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:  
*Linearity test*

Form 106-18000

80-003057  
p. 4 of 9  
*JK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      12:51

Test	AC	Time
01 Room Air	0.000	12:52
02 *Subject Test	INT*	12:52
03 Room Air	0.000	12:53

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 35  
Expiration Date = 09/03/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check*

Form 106-I8000

80-003057  
P 5 of 9  
*pm*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/06/2009      15:46

Test	AC	Time
01 Diagnostic	OK	15:47
02 Room Air	RFI*	15:47
03 Room Air	0.000	15:48

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 35  
Expiration Date = 09/03/2011  
County = 08      Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-003057  
P. 6 of 9

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      12:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:54
02 Std. Gas	0.082	12:55
03 Room Air	0.000	12:55
04 Std. Gas	0.082	12:56
05 Room Air	0.000	12:56
06 Std. Gas	0.082	12:56
07 Room Air	0.000	12:57

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-003057  
P. 7 of 9 *JK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      12:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:58
02 Std. Gas	0.081	12:58
03 Room Air	0.000	12:59
04 Std. Gas	0.082	12:59
05 Room Air	0.000	12:59
06 Std. Gas	0.082	13:00
07 Room Air	0.000	13:00

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-003057  
P. 849 *PK*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003057  
Location = TOXL      8164.13.00 06/09  
11/09/2009      13:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:01
02 Std. Gas	0.081	13:02
03 Room Air	0.000	13:02
04 Std. Gas	0.081	13:02
05 Room Air	0.000	13:03
06 Std. Gas	0.082	13:03
07 Room Air	0.000	13:04

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Celebration check*

Form 106-I8000

80-003057  
p. 9 of 9