

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004206

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR 5144 Lot No. Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 5187 Lot No. Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR 5114 Lot No. Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659357</u>	Cyl No. <u>11</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: Annual inspection, new software 8164.13

12 NOV '09
Date
1 Dec 09
Date

Deb Kashner
Field Inspector's Signature
Mary Pearson
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 13:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Sol.	0.017	13:58
03 Room Air	0.000	13:59
04 Std. Sol.	0.017	13:59
05 Room Air	0.000	14:00
06 Std. Sol.	0.018	14:01
07 Room Air	0.000	14:01

08 Sim Temp = 34.0°C

Simul Ser No = ~~0000~~ DR 5144

Std Sol No = ~~000~~ 0.019 DK

County = 08

Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-18000

*80-604206
P. 3 of 9*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 15:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Sol.	0.281	15:25
03 Room Air	0.000	15:25
04 Std. Sol.	0.286	15:26
05 Room Air	0.000	15:27
06 Std. Sol.	0.289	15:27
07 Room Air	0.000	15:28

08 Sim Temp = 34.0°C

Simul Ser No = 0000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-I8000

80-004206
p. 4 of 9 fm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 15:35

Test	AC	Time
01 Room Air	0.000	15:35
02 *Subject Test	INT*	15:36
03 Room Air	0.000	15:36

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = ~~0000~~ DR5114
Std Sol No = ~~000-0.100~~ DIC
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check

Form 106-I8000

80-004206
P. 5 of 9
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 16:08

Test	AC	Time
01 Diagnostic	OK	16:09
02 Room Air	RFI*	16:09
03 Room Air	0.000	16:10

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = ~~658357~~ 659357 016
Cyl No = 11
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-18000

80-004206
P. 6 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 16:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:14
02 Std. Gas	0.080	16:14
03 Room Air	0.000	16:15
04 Std. Gas	0.080	16:15
05 Room Air	0.000	16:16
06 Std. Gas	0.079	16:16
07 Room Air	0.000	16:17

Lot No = 659357
Cyl No = 11
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-604206
p. 7 of 9
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 16:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:18
02 Std. Gas	0.080	16:19
03 Room Air	0.000	16:19
04 Std. Gas	0.080	16:20
05 Room Air	0.000	16:20
06 Std. Gas	0.080	16:21
07 Room Air	0.000	16:21

Lot No = 659357

Cyl No = 11

Exp Date = 09/03/2011

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004206
P. 8 of 9
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
11/12/2009 16:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:22
02 Std. Gas	0.081	16:22
03 Room Air	0.000	16:23
04 Std. Gas	0.080	16:23
05 Room Air	0.000	16:24
06 Std. Gas	0.081	16:24
07 Room Air	0.000	16:24

Lot No = 659357
Cyl No = 11
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-009206
p. 9 of 9
JKM