

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004186

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. — Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. — Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5114 Lot No. — Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>41</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.083</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.083</u>	Test 5 <u>0.083</u>	Test 8 <u>0.083</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

L. Remarks/Maintenance record: New software 8164.13. Annual inspection

13 NOV 2009
Date
1 Dec 2009
Date

Deb Kashur
Field Inspector's Signature
Mary Pearson
Reviewed by

80-004186
Plaf. 7 FA

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/13/2009 09:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:44
02 Std. Sol.	0.017	09:45
03 Room Air	0.000	09:45
04 Std. Sol.	0.017	09:46
05 Room Air	0.000	09:47
06 Std. Sol.	0.017	09:47
07 Room Air	0.000	09:48

08 Sim Temp = 34.0°C

Simul Ser No = ^{DR5144} ~~00000~~ DK
Std Sol No = -000- 0.019
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-18000

80-004186
P. 3 of 9 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/13/2009 09:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:36
02 Std. Sol.	0.287	09:38
03 Room Air	0.000	09:38
04 Std. Sol.	0.289	09:39
05 Room Air	0.000	09:39
06 Std. Sol.	0.290	09:40
07 Room Air	0.000	09:41

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-18000

80-004186
p. 4 of 9 JRM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/13/2009 10:32

Test	AC	Time
01 Room Air	0.000	10:33
02 *Subject Test	INT*	10:33
03 Room Air	0.000	10:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 41
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-I8000

80-004186
P. Saf 9 ym

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/12/2009 16:31

Test	AC	Time
01 Diagnostic	OK	16:32
02 Room Air	RFI*	16:32
03 Room Air	0.000	16:33

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 41
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004186
P. Cont 9 *AK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/12/2009 14:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.083	14:30
03 Room Air	0.000	14:30
04 Std. Gas	0.083	14:30
05 Room Air	0.000	14:31
06 Std. Gas	0.082	14:31
07 Room Air	0.000	14:32

Lot No = 659358
Cyl No = 41
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:
Calibration check

Form 106-18000

80-004186
p. 7 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/12/2009 14:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:34
02 Std. Gas	0.082	14:34
03 Room Air	0.000	14:35
04 Std. Gas	0.083	14:35
05 Room Air	0.000	14:36
06 Std. Gas	0.082	14:36
07 Room Air	0.000	14:37

Lot No = 659358
Cyl No = 41
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004186
P 8 of 9 SAH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = TOXL 8164.13.00 06/09
11/12/2009 14:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:40
02 Std. Gas	0.082	14:40
03 Room Air	0.000	14:40
04 Std. Gas	0.083	14:41
05 Room Air	0.000	14:41
06 Std. Gas	0.082	14:42
07 Room Air	0.000	14:42

Lot No = 659358
Cyl No = 41
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004186
P. 9 of 9 JKM