

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003066

Location: TOXL ✓

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.) ✓
Sim SN DR5114 Lot No. Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>46</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.082</u>	Test 4 <u>0.083</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.083</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.083</u>
Average <u>0.082</u>		

L. Remarks/Maintenance record: Software 8164.13. Annual inspection

18 NOV '09
Date

Deb Kashur
Field Inspector's Signature

30 Nov 09
Date

Mary Pearson
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/18/2009 10:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Sol.	0.019	10:37
03 Room Air	0.000	10:37
04 Std. Sol.	0.020	10:38
05 Room Air	0.000	10:38
06 Std. Sol.	0.019	10:39
07 Room Air	0.000	10:40

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

low ac 0.019 ac

Form 106-18000

*80-003066
p. 3 of 9 km*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/18/2009 10:55

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Sol.	0.279	10:56
03 Room Air	0.000	10:57
04 Std. Sol.	0.282	10:57
05 Room Air	0.000	10:58
06 Std. Sol.	0.283	10:59
07 Room Air	0.000	10:59

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

linearity test 0.285 AC
~~0.028 DK~~

Form 106-I8000

80-003066
p. 4 of 9
JKL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/18/2009 13:44

Test	AC	Time
01 Room Air	0.000	13:45
02 *Subject Test	INT*	13:45
03 Room Air	0.000	13:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 46
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Interferent Test

Form 106-I8000

80-003066
P. 5 of 9
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/18/2009 13:41

Test	AC	Time
01 Diagnostic	OK	13:42
02 Room Air	RFI*	13:42
03 Room Air	0.000	13:43

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-18000

80-003066
P. Good 9/1/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/17/2009 16:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:10
02 Std. Gas	0.082	16:11
03 Room Air	0.000	16:11
04 Std. Gas	0.082	16:12
05 Room Air	0.000	16:12
06 Std. Gas	0.082	16:12
07 Room Air	0.000	16:13

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003066
P. 7 of 9 JAK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/17/2009 16:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:14
02 Std. Gas	0.083	16:14
03 Room Air	0.000	16:15
04 Std. Gas	0.083	16:15
05 Room Air	0.000	16:15
06 Std. Gas	0.083	16:16
07 Room Air	0.000	16:16

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003066
P. Sat 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.13.00 06/09
11/17/2009 16:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:17
02 Std. Gas	0.082	16:17
03 Room Air	0.000	16:18
04 Std. Gas	0.082	16:18
05 Room Air	0.000	16:19
06 Std. Gas	0.083	16:19
07 Room Air	0.000	16:20

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003066
p. 9 of 9 AK