



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number <i>80-003065</i>	Instrument Location <i>Grand Forks County Corrections</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>11/19/15</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>24 NOV 15</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
11/19/2015 01:17

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 52
Standard Expiration? 07/05/2016
Oper No? 020836

Flow Cal. Date: 04/08/2008
Slope 694
Intercept -539075

IR Calibration Date: 06/05/2008

	3um	9um
0th Coef(*100):	-9383	-9235
1st Coef(*100):	288128	138520
2nd Coef(*100):	957	781
H2O adj(mg/l*10k):	391	239

***** Printer Test End *****


Operator Signature
DAVID STROMBERG

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
11/19/2015 01:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:30
02 Std. Gas	0.081	01:30
03 Room Air	0.000	01:31
04 Std. Gas	0.081	01:31
05 Room Air	0.000	01:32
06 Std. Gas	0.080	01:32
07 Room Air	0.000	01:33

Lot No = 14214080A6
Cyl No = 52
Exp Date = 07/05/2016
County = 18 Oper No. = 100452



Operator Signature
NATHAN MOEN

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
11/19/2015 01:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:34
02 Reference	RFI*	01:34
03 Room Air	0.000	01:34

*Invalid Test
Inhibited - RFI

Lot No = 14214080A6
Cyl No = 52
Exp Date = 07/05/2016
County = 18 Oper No. = 020836

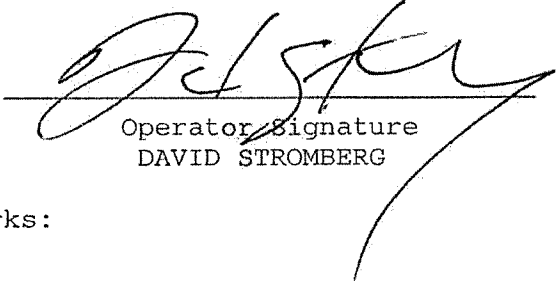

Operator Signature
DAVID STROMBERG

Remarks: