

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003060

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. Rep AC 0.019
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. Rep AC 0.285
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5114 Lot No. Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 459358 Cyl No. 17 Exp. Date 03 Sept 09
Test 1 0.082 Test 4 0.082 Test 7 0.081
Test 2 0.081 Test 5 0.081 Test 8 0.082
Test 3 0.081 Test 6 0.082 Test 9 0.081
Average 0.081

L. Remarks/Maintenance record: Software 8164.13 installed

20 NOV 2009
Date

Dob Kashur
Field Inspector's Signature

1 Dec 2009
Date

Mary Pearson
Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/19/2009 16:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&* ()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&* ()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 31
Standard Expiration? 09/03/2011
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008
 3um 9um

0th Coef(*100): -5190 -7737
1st Coef(*100): 270580 138779
2nd Coef(*100): 2499 956
H2O adj(mg/l*10k): 377 188

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print Test

Form 106-I8000

80-003060
P. def. 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/20/2009 09:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:12
02 Std. Sol.	0.020	09:13
03 Room Air	0.000	09:14
04 Std. Sol.	0.020	09:14
05 Room Air	0.000	09:15
06 Std. Sol.	0.020	09:15
07 Room Air	0.000	09:16
08 Sim Temp = 34.0°C		

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.019 AC

Form 106-I8000

80-003060
P. 3 of 9
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/20/2009 09:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:21
02 Std. Sol.	0.279	09:22
03 Room Air	0.000	09:22
04 Std. Sol.	0.283	09:23
05 Room Air	0.000	09:24
06 Std. Sol.	0.286	09:24
07 Room Air	0.000	09:25
08 Sim Temp = 34.0°C		

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-I8000

80-003060
R 4 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/20/2009 11:16

Test	AC	Time
01 Room Air	0.000	11:17
02 *Subject Test	INT*	11:17
03 Room Air	0.000	11:18

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-18000

80-003060
P. 5 of 9
11/20

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/20/2009 11:41

Test	AC	Time
01 Diagnostic	OK	11:42
02 Room Air	RFI*	11:42
03 Room Air	0.000	11:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI Test

Form 106-18000

80-003060
P. Leaf 9/11

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/19/2009 16:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.082	16:37
03 Room Air	0.000	16:37
04 Std. Gas	0.081	16:38
05 Room Air	0.000	16:38
06 Std. Gas	0.081	16:38
07 Room Air	0.000	16:39

Lot No = 659358
Cyl No = 17
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003060
p. 7 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/19/2009 16:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:41
02 Std. Gas	0.082	16:41
03 Room Air	0.000	16:41
04 Std. Gas	0.081	16:42
05 Room Air	0.000	16:42
06 Std. Gas	0.082	16:43
07 Room Air	0.000	16:43

Lot No = 659358
Cyl No = 17
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003060
p. 8 of 9, etc

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
11/19/2009 16:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:44
02 Std. Gas	0.081	16:44
03 Room Air	0.000	16:45
04 Std. Gas	0.082	16:45
05 Room Air	0.000	16:46
06 Std. Gas	0.081	16:46
07 Room Air	0.000	16:47

Lot No = 659358

Cyl No = 17

Exp Date = 09/03/2011

County = 08

Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003060
P. 9 of 9 Jan