

INTOXILYZER 8000
REPAIR AND MAINTENANCE

Serial No.: 80-004197

Location: TOXL

- | | Check When Done |
|--|-----------------|
| 1. Is a three-pronged grounded outlet used? | <u>✓</u> |
| 2. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| 3. Is the breath tube heated? | <u>✓</u> |
| 4. Is the diagnostic check complete? | <u>✓</u> |
| 5. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| 6. Print test. (Attach test record.) | <u>✓</u> |
| 7. Calibration Check. Use Ethanol Breath Standard (EBS). Do three sets in ACA mode. (Attach test records.) <u>(0.080 AC)</u> | |

Lot No. 23411080A1 Cylinder No. 14 Expir. Date 10/1/2013

Test 1 0.082 Test 4 0.081 Test 7 0.081

Test 2 0.081 Test 5 0.082 Test 8 0.081

Test 3 0.081 Test 6 0.081 Test 9 0.080

Average 0.081

- | | |
|--|----------|
| 8. Low AC. Use < 0.03 AC in ACA mode. (Attach test record.) | <u>✓</u> |
| 9. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) | <u>✓</u> |
| 10. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) | <u>✓</u> |
| 11. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) | <u>✓</u> |

Electrical Repairs:

1. Was the calibration adjusted/performed? (Attach test record.) NO
2. Was the light source changed? NO
3. Was the sample chamber cleaned? NO

Other Maintenance/Repairs:

1. Was the printer cleaned/repared? No
2. Was the solenoid cleaned? No
3. Was new tubing installed? No
4. Were additional parts replaced/changed/repared? No
5. Remarks/Maintenance record: OPERATIONAL CHECKOUT

AFTER BEING RETURNED FROM FIELD SERVICE.
READY TO GO OUT TO ANOTHER AGENCY.

21 NOV 12
Date

21 NOV 12
Date

Charles E. Eden
Field Inspector's Signature

Deb Kashner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:13
02 Std. Gas	0.082	14:13
03 Room Air	0.000	14:14
04 Std. Gas	0.081	14:14
05 Room Air	0.000	14:14
06 Std. Gas	0.081	14:15
07 Room Air	0.000	14:15

Lot No = 23411080A1
Cyl No = 14
Exp Date = 10/01/2013
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:16
02 Std. Gas	0.081	14:17
03 Room Air	0.000	14:17
04 Std. Gas	0.082	14:18
05 Room Air	0.000	14:18
06 Std. Gas	0.081	14:18
07 Room Air	0.000	14:19

Lot No = 23411080A1
Cyl No = 14
Exp Date = 10/01/2013
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Gas	0.081	14:20
03 Room Air	0.000	14:21
04 Std. Gas	0.081	14:21
05 Room Air	0.000	14:22
06 Std. Gas	0.080	14:22
07 Room Air	0.000	14:22

Lot No = 23411080A1
Cyl No = 14
Exp Date = 10/01/2013
County = 08 Oper No. = 666666

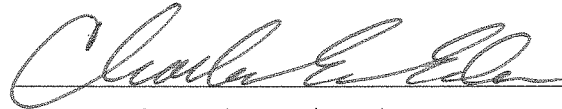

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:25
02 Std. Sol.	0.023	14:26
03 Room Air	0.000	14:26
04 Std. Sol.	0.023	14:27
05 Room Air	0.000	14:28
06 Std. Sol.	0.023	14:28
07 Room Air	0.000	14:29

08 Sim Temp = 34.0°C

Simul Ser No = DR3455

Std Sol No = 12010

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Exp. 1/4/14

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Sol.	0.288	14:32
03 Room Air	0.000	14:33
04 Std. Sol.	0.290	14:33
05 Room Air	0.000	14:34
06 Std. Sol.	0.290	14:35
07 Room Air	0.000	14:35


08 Sim Temp = 34.0°C

Simul Ser No = DR3847

Std Sol No = 12050

County = 08

Oper No. = 666666

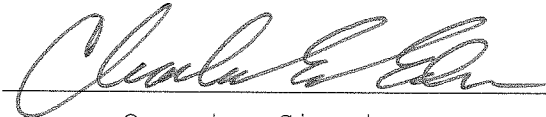

Operator Signature
CHARLES EDER

Remarks: ~~CA~~^{CS} LINEARITY TEST

0.300 AC Exp. 3/21/14

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.13.00 06/09
11/21/2012 14:38

Test	AC	Time
01 Room Air	0.000	14:39
02 *Subject Test	INT*	14:39
03 Room Air	0.000	14:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = QWERTY123456789
Dr. Lic. = ND/DIS821456
Lot No = 23411080A1
Cyl No = 14
Expiration Date = 10/01/2013
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.05% ACETONE IN ^{CEE} ~~G-DIAC~~
0.100

