



**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004205

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No.      Rep AC      ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No.      Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No.      Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓

K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>35</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.083</u>	Test 4 <u>0.083</u>	Test 7 <u>0.083</u>
Test 2 <u>0.083</u>	Test 5 <u>0.083</u>	Test 8 <u>0.082</u>
Test 3 <u>0.083</u>	Test 6 <u>0.082</u>	Test 9 <u>0.083</u>

Average                     

L. Remarks/Maintenance record: Software upgrade 8164.B. Annual inspection.

03 Dec 2009  
Date

Deb Kashur  
Field Inspector's Signature

4 Dec 2009  
Date

Mary Pearson  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/02/2009      11:34

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*\_()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^\*\_()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      659358  
Standard Cyl #?      35  
Standard Expiration?      09/03/2011  
Oper No?      777777

Flow Cal. Date:      06/04/2009  
Slope      704  
Intercept      -610062

IR Calibration Date:      06/03/2009

	3um	9um
0th Coef(*100):	-16944	-29673
1st Coef(*100):	265939	132317
2nd Coef(*100):	1696	900
H2O adj(mg/l*10k):	490	583

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      14:18

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:18
02 Std. Sol.	0.017	14:19
03 Room Air	0.000	14:20
04 Std. Sol.	0.017	14:21
05 Room Air	0.000	14:21
06 Std. Sol.	0.017	14:22
07 Room Air	0.000	14:22

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

*Low AC 0.019 AC*

Form 106-I8000

80-004205  
p. 3 of 7 km

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      14:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:43
02 Std. Sol.	0.289	14:44
03 Room Air	0.000	14:45
04 Std. Sol.	0.289	14:45
05 Room Air	0.000	14:46
06 Std. Sol.	0.289	14:47
07 Room Air	0.000	14:47

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-18000

80-004205  
P 4/7 YHL

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      14:49

Test	AC	Time
01 Room Air	0.000	14:49
02 *Subject Test	INT*	14:50
03 Room Air	0.000	14:51

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = 00000  
Std Sol No = 000  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Interferent test*

Form 106-I8000

80-004205  
P. 5/29 pm

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004205  
Location = TOXL                    8164.13.00 06/09  
12/03/2009                            10:33

Test	AC	Time
01 Diagnostic	OK	10:34
02 Room Air	RFI*	10:34
03 Room Air	0.000	10:34

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 35  
Expiration Date = 09/03/2011  
County = 08                          Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-18000

*80-004205  
P 6 of 9 1/14*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      16:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:45
02 Std. Gas	0.083	16:45
03 Room Air	0.000	16:46
04 Std. Gas	0.083	16:46
05 Room Air	0.000	16:47
06 Std. Gas	0.083	16:47
07 Room Air	0.000	16:48

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-004205  
P. 7 of 9 AM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      16:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:48
02 Std. Gas	0.083	16:49
03 Room Air	0.000	16:49
04 Std. Gas	0.083	16:50
05 Room Air	0.000	16:50
06 Std. Gas	0.082	16:51
07 Room Air	0.000	16:51

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-18000

80-004205  
P. 8 of 9 AL



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.13.00 06/09  
12/03/2009      16:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:52
02 Std. Gas	0.083	16:53
03 Room Air	0.000	16:53
04 Std. Gas	0.082	16:54
05 Room Air	0.000	16:54
06 Std. Gas	0.083	16:55
07 Room Air	0.000	16:55

Lot No = 659358  
Cyl No = 35  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-18000

80-004205  
P 9/9/09 KLM