

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program

Form: BrF-104-G

INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT

Serial No.: 80-603061 Instrument Location: STAN

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
X Other (specify) Re-install after other Intox. failed to enter ACA mode

Check When Done:

- X A. Surge Protector Installed/Properly Grounded.
- n/a B. Telephone Line Connected to Intoxilyzer® KB.
- X C. Breath Tube Heated.
- X D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- X E. Ethanol Breath Standard: _____ Connected X Installed.
(Include Form 105-G if New Cylinder Installed.)
- X F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- X G. Print Test (Level 1, Function P). (Attach Test Record.)
- X H. ACA Test (Level 1, Function C). (Attach Test Record.)
- X I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

12/07/09
11/14/09
Date
07 Dec 09
Date

[Signature]
Field Inspector's Signature
Deb Kasher
Reviewed By

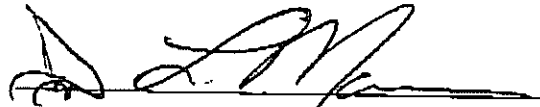
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location - STAN 8164.12.00 06/09
12/07/2009 15:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:38
02 Std. Gas	0.082	15:39
03 Room Air	0.000	15:39
04 Std. Gas	0.084	15:40
05 Room Air	0.000	15:40
06 Std. Gas	0.085	15:41
07 Room Air	0.000	15:42

Lot No - 659358
Cyl No = 43
Exp Date - 09/03/2011
County = 31 Oper No. = 100019


Operator Signature
DUANE L. MARMON

Remarks:

Form 106-18000

80-003061
P. 2 of 5
FM

**INTOXILYZER® 5000 G AND 8000
INSTALLATION AND REPAIR CHECKOUT**

Serial No.: 80-603061 Instrument Location: STAN

Reason for Install/Repair: _____ Install After Lab Repair/Inspection
X Other (specify) Re-install after other Intox. failed to enter ACA mode

Check When Done:

- X A. Surge Protector Installed/Properly Grounded.
- n/a B. Telephone Line Connected to Intoxilyzer® KB.
- X C. Breath Tube Heated.
- X D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- X E. Ethanol Breath Standard: _____ Connected X Installed.
(Include Form 105-G if New Cylinder Installed.)
- X F. Review/Enter Gas Setup (Level 1, Function S). (Intoxilyzer® 8000 Only.)
- X G. Print Test (Level 1, Function P). (Attach Test Record.)
- X H. ACA Test (Level 1, Function C). (Attach Test Record.)
- X I. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- J. Repair and/or Maintenance Performed: _____

11/14/09
Date
07 Dec 09 DK
Date

[Signature]
Field Inspector's Signature
Deb Keshu DK
Reviewed By

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = STAN 8164.12.00 06/09
11/14/2009 23:05

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#%&^*()_ + ?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#%&^*()_ + ?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 43
Standard Expiration? 09/03/2011
Oper No? 100019

Flow Cal. Date: 06/15/2009
Slope 737
Intercept -507305

IR Calibration Date: 06/04/2008
3um 9um

0th Coef(*100): -10373 -9786
1st Coef(*100): 269813 141172
2nd Coef(*100): 1454 683
H2O adj(mg/l*10k): 474 231

***** Printer Test End *****



Operator Signature
DUANE L. MARMON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = STAN 8164.12.00 06/09
11/14/2009 23:13

Test	AC	Time
01 Diagnostic	OK	23:15
02 Room Air	RFI*	23:15
03 Room Air	PUR**	23:16

*Invalid Test
 Inhibited - RFI
**Purge Fail

Sub Name = TEST, SUBJECT P
Sub DOB = 12/25/1967
Sub Sex = Female Weight = 550
Test = OTH Cit = 123456
Dr. Lic. = ND/PIG671234
Lot No = 659358
Cyl No = 43
Expiration Date = 09/03/2011
County = 31 Oper No. = 100019

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DUANE L. MARMON

Remarks:

80-003061
P. Saf S
FIL