



INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003058

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5140 Lot No. — Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓  
Sim SN DR5113 Lot No. — Rep AC 0.285
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓  
Sim SN DR5135 Lot No. — Rep AC 0.10AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 659358 Cyl No. 41 Exp. Date 9/03/2011  
Test 1 0.082 Test 4 0.083 Test 7 0.083  
Test 2 0.082 Test 5 0.083 Test 8 0.084  
Test 3 0.082 Test 6 0.083 Test 9 0.083  
Average 0.083

L. Remarks/Maintenance record: Annual Insp Software 8164.13 (GH, \*F on 12-08-09) <sup>MAP</sup>  
8 December 2009 Mandy Pearson  
<sup>MAP</sup> Date Field Inspector's Signature  
08 Dec 2009 Deb Kashur  
Date Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                8164.13.00 06/09  
12/03/2009                      10:37

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

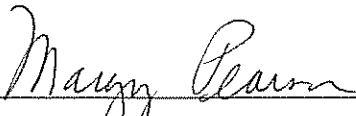
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            659358  
Standard Cyl #?            41  
Standard Expiration?       09/03/2011  
Oper No?                    107501

Flow Cal. Date:            04/07/2008  
Slope                        698  
Intercept                   -678970

IR Calibration Date:        06/05/2008  
                                        3um    9um

-----  
0th Coef(\*100):            -18867    -13178  
1st Coef(\*100):            274682    135660  
2nd Coef(\*100):            1139    678  
H2O adj(mg/l\*10k):        434    216

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
-----  
Operator Signature  
MARGY PEARSON

Remarks:

*Print*

Form 106-I8000

80-003058  
P. 2 of 9 pm

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/08/2009      12:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:08
02 Std. Sol.	0.018	12:09
03 Room Air	0.000	12:09
04 Std. Sol.	0.019	12:10
05 Room Air	0.000	12:10
06 Std. Sol.	0.019	12:11
07 Room Air	0.000	12:12

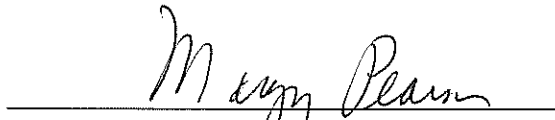
08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 00000

County = 08

Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

Low AC 0.021AC

Form 106-I8000

80-003058  
P. 3 of 9 KM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/08/2009      12:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:14
02 Std. Sol.	0.294	12:14
03 Room Air	0.000	12:15
04 Std. Sol.	0.299	12:16
05 Room Air	0.000	12:16
06 Std. Sol.	0.298	12:17
07 Room Air	0.000	12:17

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 00000

County = 08

Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

Linearity 0.285 AC

Form 106-I8000

80-003058  
P 4 of 7 KM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003058  
Location = TOXL                    8164.13.00 06/09  
12/08/2009                            12:17

Test	AC	Time
01 Room Air	0.000	12:19
02 *Subject Test	INT*	12:20
03 Room Air	0.000	12:21

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 000000  
Std Sol No = 00000  
County = 08                            Oper No. = 107501



Operator Signature  
MARGY PEARSON

Remarks:

*Interferent AC + Acetone*

Form 106-I8000

80-003058  
P. 5 of 9 Jan

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

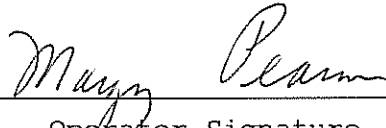
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/03/2009      11:00

Test	AC	Time
01 Diagnostic	OK	11:01
02 Room Air	RFI*	11:01
03 Room Air	0.000	11:01

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 659358  
Cyl No = 41  
Expiration Date = 09/03/2011  
County = 08      Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-003058  
P. 6 of 7 1/24

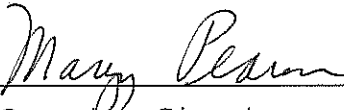
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/03/2009      11:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:17
02 Std. Gas	0.083	11:18
03 Room Air	0.000	11:18
04 Std. Gas	0.083	11:19
05 Room Air	0.000	11:19
06 Std. Gas	0.083	11:19
07 Room Air	0.000	11:20

Lot No = 659358  
Cyl No = 41  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

AC

Form 106-I8000

80-003058  
P 7 of 9 1/2

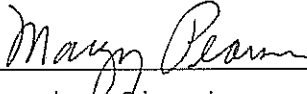
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/03/2009      11:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:13
02 Std. Gas	0.082	11:13
03 Room Air	0.000	11:14
04 Std. Gas	0.082	11:14
05 Room Air	0.000	11:15
06 Std. Gas	0.082	11:15
07 Room Air	0.000	11:16

Lot No = 659358  
Cyl No = 41  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003058  
P. 8 of 9 FM



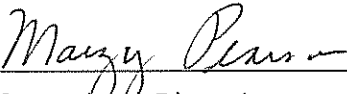
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = TOXL      8164.13.00 06/09  
12/03/2009      11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.083	11:22
03 Room Air	0.000	11:22
04 Std. Gas	0.084	11:23
05 Room Air	0.000	11:23
06 Std. Gas	0.083	11:23
07 Room Air	0.000	11:24

Lot No = 659358  
Cyl No = 41  
Exp Date = 09/03/2011  
County = 08      Oper No. = 107501

  
\_\_\_\_\_  
Operator Signature  
MARGY PEARSON

Remarks:

ACA

Form 106-I8000

80-003058  
P. Sep 9 11:21