

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004199

Location: TOX LAB

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5190 Lot No. — Rep AC 0.021 AC ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.285 AC ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10 AC ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>41</u>	Exp. Date <u>9/03/2011</u>
Test 1 <u>0.083</u>	Test 4 <u>0.083</u>	Test 7 <u>0.083</u>
Test 2 <u>0.083</u>	Test 5 <u>0.083</u>	Test 8 <u>0.083</u>
Test 3 <u>0.083</u>	Test 6 <u>0.083</u>	Test 9 <u>0.083</u>
Average <u>0.083</u>		

L. Remarks/Maintenance record: Software 8164.13 (Annual Inspec)

<u>8 December 2009</u> Date	<u>Mary Person</u> Field Inspector's Signature
<u>08 Dec 2009</u> Date	<u>Deb Kashur</u> Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 08:12

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 19
Standard Expiration? 09/03/2011
Oper No? 107501

Flow Cal. Date: 06/04/2009
Slope 701
Intercept -580043

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100): -44451 -27073
1st Coef(*100): 294251 132474
2nd Coef(*100): -1375 1189
H2O adj(mg/l*10k): 315 545

***** Printer Test End *****

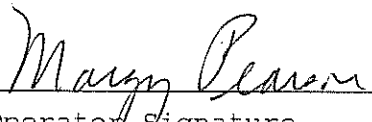

Operator Signature
MARGY PEARSON

Remarks:

Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 08:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:38
02 Std. Sol.	0.018	08:40
03 Room Air	0.000	08:40
04 Std. Sol.	0.018	08:41
05 Room Air	0.000	08:41
06 Std. Sol.	0.019	08:42
07 Room Air	0.000	08:43

08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 08

Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Low AC 0.021 AC

Form 106-I8000

*80-004199
P. 3 of 9 PL*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 08:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:49
02 Std. Sol.	0.291	08:50
03 Room Air	0.000	08:50
04 Std. Sol.	0.292	08:51
05 Room Air	0.000	08:51
06 Std. Sol.	0.294	08:52
07 Room Air	0.000	08:53


08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 08

Oper No. = 107501

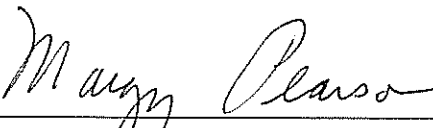

Operator Signature
MARGY PEARSON

Remarks:

Linearity 0.285 AC

Form 106-I8000

80-004199
P. 4-9-09


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 09:08

Test	AC	Time
01 Room Air	0.000	09:09
02 *Subject Test	INT*	09:09
03 Room Air	0.000	09:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 08 Oper No. = 107501

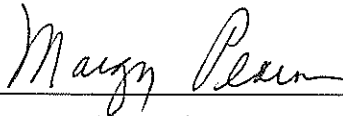

Operator Signature
MARGY PEARSON

Remarks:

Interferent - ALC + AC

Form 106-18000

80-004199
P. 5 of 9/12/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 09:06

Test	AC	Time
01 Diagnostic	OK	09:07
02 Room Air	RFI*	09:07
03 Room Air	0.000	09:08

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

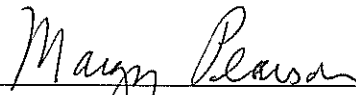

Operator Signature
MARGY PEARSON

Remarks:

RFI Test

Form 106-I8000

80-004199
P. 6 of 9 JAH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Gas	0.083	09:19
03 Room Air	0.000	09:20
04 Std. Gas	0.083	09:20
05 Room Air	0.000	09:21
06 Std. Gas	0.083	09:21
07 Room Air	0.000	09:21

Lot No = 659358
Cyl No = 19
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

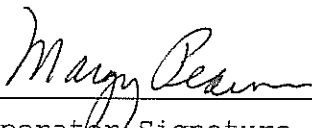

Operator Signature
MARGY PEARSON

Remarks:

ACA - Cal Check

Form 106-I8000

*80-004199
P. 7 of 9*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 09:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:25
02 Std. Gas	0.083	09:25
03 Room Air	0.000	09:26
04 Std. Gas	0.083	09:26
05 Room Air	0.000	09:27
06 Std. Gas	0.083	09:27
07 Room Air	0.000	09:27

Lot No = 659358
Cyl No = 19
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

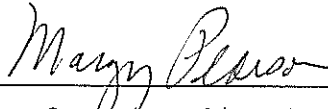

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check

Form 106-I8000

80-004199
P. 8 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.13.00 06/09
12/08/2009 09:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Gas	0.083	09:29
03 Room Air	0.000	09:30
04 Std. Gas	0.083	09:30
05 Room Air	0.000	09:31
06 Std. Gas	0.083	09:31
07 Room Air	0.000	09:31

Lot No = 659358
Cyl No = 19
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Cal Check ACA

Form 106-I8000

80-004199
P. 949 JCH