

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-002669

Location: Tox LAB

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5190 Lot No. — Rep AC 0.021 AC
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5113 Lot No. — Rep AC 0.285 AC
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + Acetone
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 659358 Cyl No. 34 Exp. Date 09/03/2011
Test 1 0.078 Test 4 0.078 Test 7 0.079
Test 2 0.078 Test 5 0.079 Test 8 0.078
Test 3 0.078 Test 6 0.078 Test 9 0.079
Average 0.078 AC

L. Remarks/Maintenance record: SOFTWARE 8164.13 (Annual Inspec)
11 Dec 09 Date Margaret A. Ryan Field Inspector's Signature
15 Dec 09 Date Deb Kashner Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:16
02 Std. Sol.	0.019	14:17
03 Room Air	0.000	14:17
04 Std. Sol.	0.019	14:18
05 Room Air	0.000	14:18
06 Std. Sol.	0.019	14:19
07 Room Air	0.000	14:20


08 Sim Temp = 34.0°C

Simul Ser No = 00000000

Std Sol No = 0000

County = 08

Oper No. = 107501

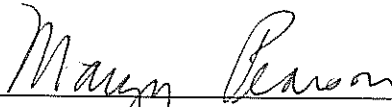

Operator Signature
MARGY PEARSON

Remarks:

0.021 AC. LOW

Form 106-I8000

80-002669
P. 3 of 9 VLN

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:33
02 Std. Sol.	0.289	14:34
03 Room Air	0.000	14:34
04 Std. Sol.	0.288	14:35
05 Room Air	0.000	14:36
06 Std. Sol.	0.288	14:36
07 Room Air	0.000	14:37


08 Sim Temp = 34.0°C

Simul Ser No = 00000000

Std Sol No = 0000

County = 08

Oper No. = 107501

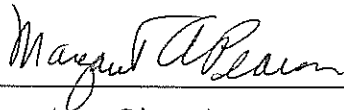

Operator Signature
MARGY PEARSON

Remarks:

Linearity

Form 106-I8000

80-002669
P. 4/8/17


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:37

Test	AC	Time
01 Room Air	0.000	14:38
02 *Subject Test	INT*	14:39
03 Room Air	0.000	14:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 00000000
Std Sol No = 0000
County = 08 Oper No. = 107501

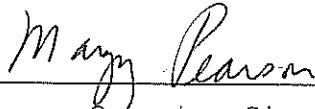

Operator Signature
MARGY PEARSON

Remarks:

Interfer - Alc and Acetone

Form 106-18000

*80-002669
P. 5 of 9/11/09*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:50

Test	AC	Time
01 Diagnostic	OK	14:51
02 Room Air	RFI*	14:52
03 Room Air	0.000	14:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 00000000
Std Sol No = 0000
County = 08 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-002669
P. 6 of 9 YH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:54
02 Std. Gas	0.078	14:55
03 Room Air	0.000	14:55
04 Std. Gas	0.078	14:56
05 Room Air	0.000	14:56
06 Std. Gas	0.078	14:57
07 Room Air	0.000	14:57

Lot No = 659358
Cyl No = 34
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

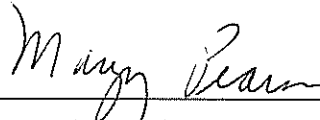

Operator Signature
MARGY PEARSON

Remarks:

Cal Check - ACA - 0.080 AC

Form 106-I8000

80-002669
P. 7/9 YH


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 14:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Gas	0.078	14:58
03 Room Air	0.000	14:59
04 Std. Gas	0.079	14:59
05 Room Air	0.000	15:00
06 Std. Gas	0.078	15:00
07 Room Air	0.000	15:01

Lot No = 659358
Cyl No = 34
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

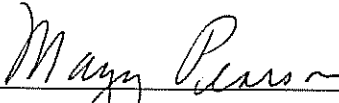

Operator Signature
MARGY PEARSON

Remarks:

ACA - Cal check

Form 106-I8000

*80-002669
P. 8 of 9*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.13.00 06/09
12/11/2009 15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.079	15:02
03 Room Air	0.000	15:03
04 Std. Gas	0.078	15:03
05 Room Air	0.000	15:03
06 Std. Gas	0.079	15:04
07 Room Air	0.000	15:04

Lot No = 659358
Cyl No = 34
Exp Date = 09/03/2011
County = 08 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

ACA - Cal Check

Form 106-I8000

80-002669
P.949 KH