



**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004194

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No.      Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No.      Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No.      Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>54</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080<sup>DK</sup> 0.079</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: Software 8164.13. Annual inspection

<u>11 Dec 2009</u> Date	<u>Deb Kashur</u> Field Inspector's Signature
<u>15 Dec 2009</u> Date	<u>Mary Pearson</u> Reviewed by

80-004194  
P. 1 of 9  
JCN

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004194  
Location = TOXL                      8164.13.00 06/09  
12/10/2009                              13:54

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            659358  
Standard Cyl #?            54  
Standard Expiration?        09/03/2011  
Oper No?                    777777

Flow Cal. Date:            06/23/2009  
Slope                        681  
Intercept                   -622430

IR Calibration Date:        12/10/2009  
                                      3um                        9um

	3um	9um
0th Coef(*100):	-30896	-34419
1st Coef(*100):	274294	133969
2nd Coef(*100):	-715	552
H2O adj(mg/l*10k):	491	645

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004194  
Location = TOXL                8164.13.00 06/09  
12/10/2009                      15:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:22
02 Std. Sol.	0.020	15:23
03 Room Air	0.000	15:23
04 Std. Sol.	0.019	15:24
05 Room Air	0.000	15:25
06 Std. Sol.	0.020	15:25
07 Room Air	0.000	15:26

08 Sim Temp = 34.0°C

Simul Ser No = 000

Std Sol No = 0000

County = 08

Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Low AC 0.019 ac*

Form 106-18000

80-004194  
P. 3 of 8 FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004194  
Location = TOXL      8164.13.00 06/09  
12/11/2009      07:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:10
02 Std. Sol.	0.289	07:11
03 Room Air	0.000	07:11
04 Std. Sol.	0.291	07:12
05 Room Air	0.000	07:12
06 Std. Sol.	0.289	07:13
07 Room Air	0.000	07:14

08 Sim Temp = 34.0°C

Simul Ser No = 000

Std Sol No = 0000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-I8000

80-004194  
P. 4-4 9 VAM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004194  
Location = TOXL                    8164.13.00 06/09  
12/11/2009                            07:28

Test	AC	Time
01 Room Air	0.000	07:29
02 *Subject Test	INT*	07:29
03 Room Air	0.000	07:30

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 54  
Expiration Date = 09/03/2011  
County = 08                            Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Interferent test*

Form 106-18000

80-004194  
P. 5 of 9 YH

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004194  
Location = TOXL                              8164.13.00 06/09  
12/11/2009                                      07:54

Test	AC	Time
01 Diagnostic	OK	07:55
02 Room Air	RFI*	07:55
03 Room Air	0.000	07:55

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI                                      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 659358  
Cyl No = 54  
Expiration Date = 09/03/2011  
County = 08                                      Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

RFI test

Form 106-18000

80-004194  
P. G. 9/1/09

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004194  
Location = TOXL                 8164.13.00 06/09  
12/11/2009                        08:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:34
02 Std. Gas	0.080	08:34
03 Room Air	0.000	08:35
04 Std. Gas	0.080	08:35
05 Room Air	0.000	08:35
06 Std. Gas	0.080	08:36
07 Room Air	0.000	08:36

Lot No = 659358  
Cyl No = 54  
Exp Date = 09/03/2011  
County = 08                        Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check      0.080 AC*

Form 106-18000

*80-004194  
P. 7 of 7 AM*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004194  
Location = TOXL                              8164.13.00 06/09  
12/11/2009    08:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:38
02 Std. Gas	0.080	08:38
03 Room Air	0.000	08:39
04 Std. Gas	0.080	08:39
05 Room Air	0.000	08:40
06 Std. Gas	0.080	08:40
07 Room Air	0.000	08:40

Lot No = 659358  
Cyl No = 54  
Exp Date = 09/03/2011  
County = 08                                      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks: Calibration check 0.080 AC

Form 106-18000

80-004194  
P. 8 of 9/AM



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer  
North Dakota Model 8000  
Location = TOXL  
12/11/2009

Alcohol Analyzer  
SN 80-004194  
8164.13.00 06/09  
08:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:43
02 Std. Gas	0.079	08:43
03 Room Air	0.000	08:44
04 Std. Gas	0.080	08:44
05 Room Air	0.000	08:45
06 Std. Gas	0.080	08:45
07 Room Air	0.000	08:46

Lot No = 659358

Cyl No = 54

Exp Date = 09/03/2011

County = 08

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*

Form 106-18000

*80-004194  
P. 9 of 9 KK*