



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <b>4199</b>	Instrument Location <b>Mott, ND 58646</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <b>12/12/11</b>
Reviewed By (Crime Laboratory Use Only) <b>Deb Kashner</b>	Date <b>03 Apr 2012</b>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004199  
Location = MOTT                8164.13.00 06/09  
12/12/2011                      14:06

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            23411080A1  
Standard Cyl #?            60  
Standard Expiration?       10/01/2013  
Oper No?                    011119

Flow Cal. Date:            07/05/2011  
Slope                      672  
Intercept                  -604945

IR Calibration Date:       07/05/2011  
                                  3um                    9um

-----  
0th Coef(\*100):        -11921                -14558  
1st Coef(\*100):        259221                127392  
2nd Coef(\*100):        3824                    1697  
H2O adj(mg/l\*10k):    486                    640

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
-----  
Operator Signature  
JOSH MONSON

Remarks:

Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004199  
Location = MOTT      8164.13.00 06/09  
12/12/2011      14:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:07
02 Std. Gas	0.083	14:08
03 Room Air	0.000	14:08
04 Std. Gas	0.083	14:08
05 Room Air	0.000	14:09
06 Std. Gas	0.083	14:09
07 Room Air	0.000	14:10

Lot No = 23411080A1  
Cyl No = 60  
Exp Date = 10/01/2013  
County = 21      Oper No. = 011119

  
Operator Signature  
JOSH MONSON

Remarks:

ACA Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

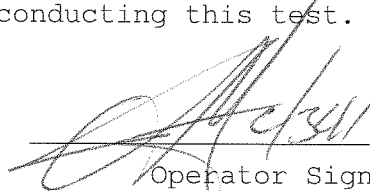
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004199  
Location = MOTT      8164.13.00 06/09  
12/12/2011      14:11

Test	AC	Time
01 Diagnostic	OK	14:12
02 Room Air	RFI*	14:12
03 Room Air	0.000	14:12

\*Invalid Test  
Inhibited - RFI

Sub Name = MONSON, JOSHUA RUSSELL  
Sub DOB = 07/02/1982  
Sub Sex = Male      Weight = 235  
Test = DUI      Cit = NA  
Dr. Lic. = ND/MON825956  
Lot No = 23411080A1  
Cyl No = 60  
Expiration Date = 10/01/2013  
County = 21      Oper No. = 011119

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
JOSH MONSON

Remarks:

*RFI Check Test*