

**INTOXILYZER 5000 G and 8000 INSPECTION**

Serial No.: 80-004200

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5144 Lot No.      Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5187 Lot No.      Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5114 Lot No.      Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 659350 Cyl No. 17 Exp. Date 03 Sept 2011

Test 1 0.080 Test 4 0.079 Test 7 0.079

Test 2 0.079 Test 5 0.079 Test 8 0.079

Test 3 0.079 Test 6 0.080 Test 9 0.079

Average 0.079

L. Remarks/Maintenance record: Software upgrade 8164.13. Annual inspection.

21 Dec 2009  
Date

Deb Kashur  
Field Inspector's Signature

30 Dec 2009  
Date

Maryann  
Reviewed by

80-004200  
P. 1 of 7 FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL              8164.13.00 06/09  
12/21/2009                              12:50

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOpqrstuvwxyz!@#%&^\* () \_ + ?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOpqrstuvwxyz!@#%&^\* () \_ + ?

Current Instrument Setup

Data Entry Mode:              Enabled  
Start Test Sequence:          DABACABA  
Display Prelim Rslt?          Yes  
Display Third Digit?          Yes  
Inhib Printer(Y/N)?          No  
Display Volume?                No  
Disable On Memfull?          Yes  
# of Print Copies?            1  
Select Std (D/W/I)?          Dry  
Standard Value?                0.080  
Standard Lot #?                659358  
Standard Cyl #?                17  
Standard Expiration?          09/03/2011  
Oper No?                        777777

Flow Cal. Date:                06/04/2009  
Slope                            715  
Intercept                       -655593

IR Calibration Date:          12/21/2009  
                                       3um                              9um

-----  
0th Coef(\*100):               -28821                              -30199  
1st Coef(\*100):               265418                              139385  
2nd Coef(\*100):               -207                                 427  
H2O adj(mg/l\*10k):          678                                 645

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004200  
P. 2 of 9 15/11

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      13:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:13
02 Std. Sol.	0.021	13:13
03 Room Air	0.000	13:14
04 Std. Sol.	0.020	13:15
05 Room Air	0.000	13:15
06 Std. Sol.	0.021	13:16
07 Room Air	0.000	13:16

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature

DEB KASHUR

Remarks:

Low AC      0.019 AC

Form 106-I8000

80-004200  
P. 3 of 9 PM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      13:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:42
02 Std. Sol.	0.287	13:43
03 Room Air	0.000	13:44
04 Std. Sol.	0.288	13:44
05 Room Air	0.000	13:45
06 Std. Sol.	0.288	13:46
07 Room Air	0.000	13:46

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*linearity test 0.285 AC*

Form 106-18000

80-004200  
P. 449 1/11

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      13:58

Test	AC	Time
01 Room Air	0.000	13:59
02 *Subject Test	INT*	13:59
03 Room Air	0.000	14:00

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = 00000  
Std Sol No = 000  
County = 08      Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Interferent test*

Form 106-I8000

80-004200  
P. 5 of 9 PM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004200  
Location = TOXL                    8164.13.00 06/09  
12/21/2009                            14:16

Test	AC	Time
01 Diagnostic	OK	14:17
02 Room Air	RFI*	14:17
03 Room Air	0.000	14:17

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Simul Ser No = 00000  
Std Sol No = 000  
County = 08                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

RFI Test

Form 106-18000

80-004200  
P. 6 of 7

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      14:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.080	14:55
03 Room Air	0.000	14:56
04 Std. Gas	0.079	14:56
05 Room Air	0.000	14:57
06 Std. Gas	0.079	14:57
07 Room Air	0.000	14:58

Lot No = 659358  
Cyl No = 17  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-004200  
P. 7 of 9 PM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.079	15:02
03 Room Air	0.000	15:03
04 Std. Gas	0.079	15:03
05 Room Air	0.000	15:03
06 Std. Gas	0.080	15:04
07 Room Air	0.000	15:04

Lot No = 659358  
Cyl No = 17  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*

Form 106-I8000

80-004200  
P. 8-19 YH



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.13.00 06/09  
12/21/2009      15:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:05
02 Std. Gas	0.079	15:05
03 Room Air	0.000	15:06
04 Std. Gas	0.079	15:06
05 Room Air	0.000	15:07
06 Std. Gas	0.079	15:07
07 Room Air	0.000	15:08

Lot No = 659358  
Cyl No = 17  
Exp Date = 09/03/2011  
County = 08      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check*  
Form 106-I8000

80-004200  
P. 9 of 9