



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (5-2011)

Chemical Test Operator Name (Print) MATTHEW ANDERSON		
Location RICHLAND COUNTY JAIL	Intoxilyzer® Serial Number 80-005944	
Gas Lot Number 29014080A2	Gas Cylinder Number 12	Gas Expiration Date 12/05/2016

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.083 AC
 - B. 0.084 AC
 - C. 0.084 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Matthew Anderson</i>	Date 12/25/2015
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date 31 Dec 2015

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 .06/09
12/25/2015 16:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:21
02 Std. Gas	0.083	16:22
03 Room Air	0.000	16:22
04 Std. Gas	0.084	16:23
05 Room Air	0.000	16:23
06 Std. Gas	0.084	16:23
07 Room Air	0.000	16:24


Lot No = 29614080A2

Cyl No = 12

Exp Date = 12/05/2016

County = 39

Oper No. = 020401

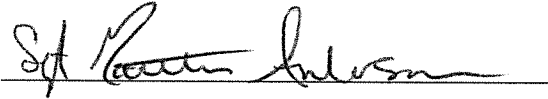

Operator Signature
MATTHEW ANDERSON

Remarks:

Form 106-I8000