



ETHANOL GAS STANDARD CYLINDER REPORT
OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Shannon Liebald</i>		
Location <i>Mercer County Jail</i>	Intoxilyzer® Serial Number <i>80-004949</i>	
Gas Lot Number <i>01316080A2</i>	Gas Cylinder Number <i>36</i>	Gas Expiration Date <i>02-05-2018</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.077* AC

B. *0.077* AC

C. *0.078* AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>Shannon Liebald</i>	Date <i>06-04-2016</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>07 June 16</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
06/04/2016 07:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:40
02 Std. Gas	0.077	07:40
03 Room Air	0.000	07:41
04 Std. Gas	0.077	07:41
05 Room Air	0.000	07:41
06 Std. Gas	0.078	07:42
07 Room Air	0.000	07:42

Lot No = 01316080A2
Cyl No = 36
Exp Date = 02/05/2018
County = 29 Oper No. = 020904



Operator Signature
SHANNON P. LIEBOLD

Remarks: