



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (10-2016)

Serial Number <del>1361508045</del> 80-005361	Instrument Location MOUNTAIN COUNTY JAIL
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P). ✓
  - B. ACA Test (Level 1, Function C). ✓
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test). ✓
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>John E. Shaw</i>	Date 10-13-16
--	------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date 14 Feb 17
--	-----------------------------



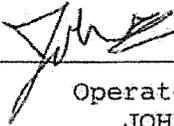
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005361  
Location = STAN      8164.14.00 09/16  
10/13/2016      21:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:48
02 Std. Gas	0.081	21:48
03 Room Air	0.000	21:49
04 Std. Gas	0.080	21:49
05 Room Air	0.000	21:50
06 Std. Gas	0.080	21:50
07 Room Air	0.000	21:50

Lot No = 19615080A5  
Cyl No = 16  
Exp Date = 09/05/2017  
County = 31      Oper No. = 031123

  
Operator Signature  
JOHN TRONSET

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005361  
Location = STAN      8164.14.00 09/16  
10/13/2016      21:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	21:46
02 Room Air	RFI*	21:46

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 16  
Exp Date = 09/05/2017  
County = 31      Oper No. = 031123

  
\_\_\_\_\_  
Operator Signature  
JOHN TRONSET

Remarks:

Form 106-I8000