



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-006497	Instrument Location Grand Forks Air Force Base
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Moved for updates / reinstallation after updates	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): **update complete**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 12/19/16
-------------------------------	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by Deb Shanaver	Certified Date 12 Jan 17
--	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
12/19/2016 16:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 30
Standard Expiration? 09/05/2017
Oper No? 020836

Flow Cal. Date: 08/19/2015
Slope 657
Intercept -642041

IR Calibration Date: 08/19/2015
 3um 9um

	3um	9um
0th Coef(*100):	-16255	-17993
1st Coef(*100):	263339	136695
2nd Coef(*100):	5198	1666
H2O adj(mg/l*10k):	766	522

***** Printer Test End *****

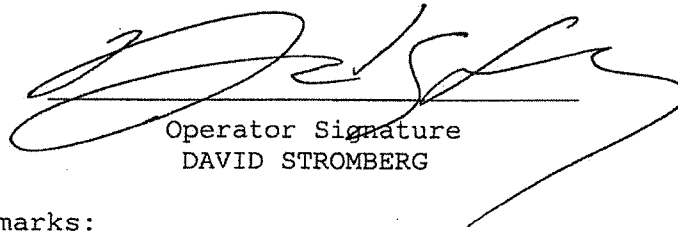

Operator Signature
DAVID STROMBERG

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAP 8164.14.00 09/16
12/19/2016 16:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:30
02 Std. Gas	0.081	16:30
03 Room Air	0.000	16:31
04 Std. Gas	0.082	16:31
05 Room Air	0.000	16:32
06 Std. Gas	0.082	16:32
07 Room Air	0.000	16:33

Lot No = 19615080A5
Cyl No = 30
Exp Date = 09/05/2017
County = 18 Oper No. = 020836

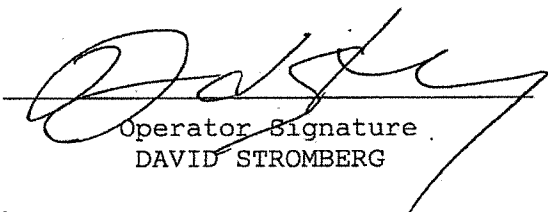
Operator Signature
DAVID STROMBERG

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58500

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAP 8164.14.00 09/16
12/19/2016 16:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	16:34
02 Room Air	0.000	16:34

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 30
Exp Date = 09/05/2017
County = 18 Oper No. = 020836


Operator Signature
DAVID STROMBERG

Remarks: