



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Hunter Lewis</i>		
Location <i>NWPO</i>	Intoxilyzer® Serial Number <i>50-00447</i>	
Gas Lot Number <i>04210080A3</i>	Gas Cylinder Number <i>416</i>	Gas Expiration Date <i>5/5/2018</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.082* AC

B. *0.083* AC

C. *0.083* AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>12/12/16</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>20 Dec 16</i>

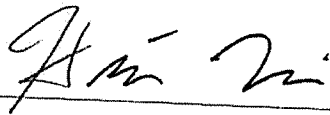
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = NAPO      8164.14.00 09/16  
12/12/2016      10:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:03
02 Std. Gas	0.082	10:04
03 Room Air	0.000	10:04
04 Std. Gas	0.083	10:05
05 Room Air	0.000	10:05
06 Std. Gas	0.083	10:05
07 Room Air	0.000	10:06

Lot No = 09216080A3  
Cyl No = 46  
Exp Date = 05/05/2018  
County = 24      Oper No. = 131282

  
\_\_\_\_\_  
Operator Signature  
HUNTER LEWIS

Remarks: