



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number <i>80-005949</i>	Instrument Location <i>BSPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Pat Reuz</i>	Date <i>12-28-16</i>
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shevaver</i>	Certified Date <i>13 Jan 17</i>
--	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005949
Location = BSPD 8164.14.00 09/16
12/28/2016 16:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 35
Standard Expiration? 05/05/2018
Oper No? 029710

Flow Cal. Date: 06/20/2013
Slope 692
Intercept -708987

IR Calibration Date: 04/06/2015
3um 9um

0th Coef(*100):	-24726	-28868
1st Coef(*100):	263448	133361
2nd Coef(*100):	3398	1497
H2O adj(mg/l*10k):	326	367

***** Printer Test End *****


Operator Signature
PATRICK RENZ

Remarks: Install Intoxilyzer


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005949
Location = BSPD 8164.14.00 09/16
12/28/2016 16:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:52
02 Std. Gas	0.078	16:52
03 Room Air	0.000	16:53
04 Std. Gas	0.077	16:53
05 Room Air	0.000	16:54
06 Std. Gas	0.078	16:54
07 Room Air	0.000	16:54

Lot No = 09216080A3
Cyl No = 35
Exp Date = 05/05/2018
County = 08 Oper No. = 029710

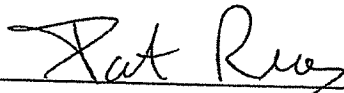

Operator Signature
PATRICK RENZ

Remarks: Install Intoxilyzer

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005949
Location = BSPD 8164.14.00 09/16
12/28/2016 16:55


DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:56
02 Std. Gas	RFI*	16:56
03 Room Air	0.000	16:57

*Invalid Test
Inhibited - RFI

Lot No = 09216080A3
Cyl No = 35
Exp Date = 05/05/2018
County = 08

Oper No. = 029710

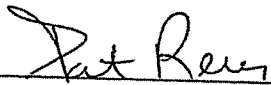
Operator Signature
PATRICK RENZ

Remarks: *Install Intoxilyzer*

Form 106-I8000