



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Chad Thompson</i>		
Location <i>Cass Co. Jail</i>		Intoxilyzer® Serial Number <i>80-006506</i>
Gas Lot Number <i>17316080A2</i>	Gas Cylinder Number <i>34</i>	Gas Expiration Date <i>8-5-18</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.081* AC
  - B. *0.081* AC
  - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>2-20-2017</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>22 Feb 17</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = CASS      8164.14.00 09/16  
02/20/2017      03:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	03:49
02 Std. Gas	0.081	03:49
03 Room Air	0.000	03:50
04 Std. Gas	0.081	03:50
05 Room Air	0.000	03:51
06 Std. Gas	0.081	03:51
07 Room Air	0.000	03:52

Lot No = 17316080A2  
Cyl No = 34  
Exp Date = 08/05/2018  
County = 09      Oper No. = 050910

 3524

Operator Signature  
CHAD THOMPSON

Remarks:

ACA Check for Cylinder  
Install

Form 106-I8000