



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-005357	Instrument Location UND-PD
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Capt. Tracy L. Meindorff</i>	Date 03-01-2017
--	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date 01 Mar 17
--	------------------------------------

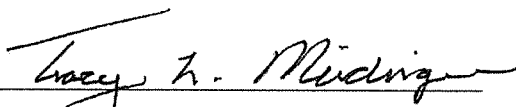
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005357
Location = UNDP 8164.14.00 09/16
03/01/2017 07:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:45
02 Std. Gas	0.081	07:46
03 Room Air	0.000	07:46
04 Std. Gas	0.081	07:46
05 Room Air	0.000	07:47
06 Std. Gas	0.082	07:47
07 Room Air	0.000	07:48

Lot No = 19615080A5
Cyl No = 21
Exp Date = 09/05/2017
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000

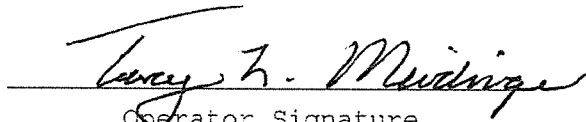
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005357
Location = UNDP 8164.14.00 09/16
03/01/2017 07:49

Test	AC	Time
01 Room Air	RFI*	07:51
02 Room Air	0.000	07:52

*Invalid Test
Inhibited - RFI

Sub Name = TEST, SUBJECT N
Sub DOB = 01/01/1967
Sub Sex = Male Weight = 195
Test = DUI Cit = 9999999
Dr. Lic. = ND/XXX457777
Lot No = 19615080A5
Cyl No = 21
Expiration Date = 09/05/2017
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000