

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004945

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3060 Lot No. 201507F Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. IC54 Solution 0.10AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 8/5/2018

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 642 psi Reg 650 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4/4/17
Date

Charles E. [Signature]
Field Inspector's Signature

04 Apr 17
Date

Deb Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 09:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 26
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 09/26/2012
Slope 622
Intercept -485836

IR Calibration Date: 04/22/2016
 3um 9um

0th Coef(*100): -23427 -26406
1st Coef(*100): 266989 144341
2nd Coef(*100): 2085 1078
H2O adj(mg/l*10k): 539 400

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 09:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:37
02 Std. Sol.	0.017	09:38
03 Room Air	0.000	09:38
04 Std. Sol.	0.018	09:39
05 Room Air	0.000	09:40
06 Std. Sol.	0.017	09:40
07 Room Air	0.000	09:41

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 201507F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low & CEE AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 09:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:55
02 Std. Sol.	0.390	09:56
03 Room Air	0.000	09:56
04 Std. Sol.	0.392	09:57
05 Room Air	0.000	09:58
06 Std. Sol.	0.392	09:58
07 Room Air	0.000	09:59

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 10:00

Test	AC	Time
01 Room Air	0.000	10:01
02 *Subject Test	INT*	10:02
03 Room Air	0.000	10:02

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

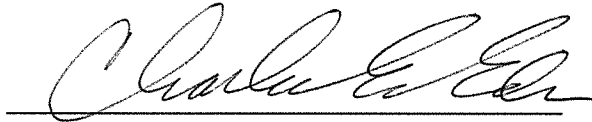

Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 10:03

Test	AC	Time
01 Diagnostic	OK	10:04
02 Room Air	0.000	10:05
03 *Subject Test	RFI*	10:05
04 Room Air	0.000	10:05

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

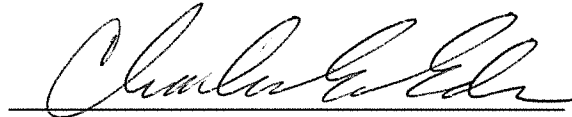

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 10:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:09
02 Std. Gas	0.080	10:09
03 Room Air	0.000	10:10
04 Std. Gas	0.080	10:10
05 Room Air	0.000	10:10
06 Std. Gas	0.080	10:11
07 Room Air	0.000	10:11

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 10:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:12
02 Std. Gas	0.080	10:13
03 Room Air	0.000	10:13
04 Std. Gas	0.081	10:13
05 Room Air	0.000	10:14
06 Std. Gas	0.081	10:14
07 Room Air	0.000	10:15

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

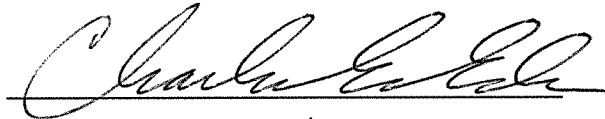

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/04/2017 10:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:15
02 Std. Gas	0.080	10:16
03 Room Air	0.000	10:16
04 Std. Gas	0.081	10:17
05 Room Air	0.000	10:17
06 Std. Gas	0.080	10:17
07 Room Air	0.000	10:18

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

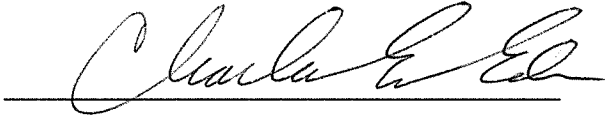

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC
Form 106-I8000