

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005938

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN ~~MP3060~~ Lot No. 201507F Rep AC 0.020  
MP3060
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3063 Lot No. 16102 Rep AC 0.400
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. 1CS4 Solution 0.10AC + 0.05% ACETONE
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 8/5/2018

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.0823</u> <sup>CEE</sup>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 633 psi Reg 650 psi

K. Remarks/Maintenance record: REPLACED GAS REGULATOR, ANNUAL INSPECTION

03 APR 2017  
Date  
04 Apr 17  
Date

Charles E. E...  
Field Inspector's Signature  
Deb Sheaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer          Alcohol Analyzer  
North Dakota Model 8000          SN 80-005938  
Location = TOXL                      8164.14.00 09/16  
04/03/2017                                  13:54

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:                      Enabled  
Start Test Sequence:                  DABACABA  
Display Prelim Rslt?                  Yes  
Display Third Digit?                  Yes  
Inhib Printer (Y/N)?                  No  
Display Volume?                        No  
Disable On Memfull?                  Yes  
# of Print Copies?                      1  
Select Std (D/W/I)?                  Dry  
Standard Value?                        0.080  
Standard Lot #?                         17316080A2  
Standard Cyl #?                        26  
Standard Expiration?                  08/05/2018  
Oper No?                                666666

Flow Cal. Date:                        07/15/2013  
Slope                                    664  
Intercept                               -578364

IR Calibration Date:                  07/27/2015

	3um	9um
0th Coef (*100):	-14232	-19463
1st Coef (*100):	289698	139293
2nd Coef (*100):	2380	1071
H2O adj (mg/l*10k):	253	294

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005938  
Location = TOXL      8164.14.00 09/16  
04/03/2017      13:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Sol.	0.020	13:57
03 Room Air	0.000	13:58
04 Std. Sol.	0.019	13:59
05 Room Air	0.000	13:59
06 Std. Sol.	0.020	14:00
07 Room Air	0.000	14:00

08 Sim Temp = 34.0°C

Simul Ser No = MP3060  
Std Sol No = 201507F  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005938  
Location = TOXL                 8164.14.00 09/16  
04/03/2017                        14:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Sol.	0.393	14:12
03 Room Air	0.000	14:13
04 Std. Sol.	0.395	14:13
05 Room Air	0.000	14:14
06 Std. Sol.	0.395	14:15
07 Room Air	0.000	14:15

08 Sim Temp = 34.0°C

Simul Ser No = MP3063  
Std Sol No = 16102  
County = 08                            Oper No. = 666666

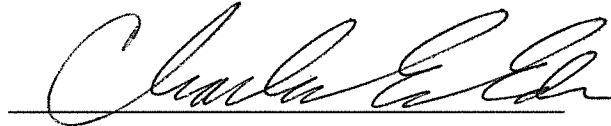

Operator Signature  
CHARLES EDER

Remarks:

*HIGH AC*  
*0.400 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005938  
Location = TOXL      8164.14.00 09/16  
04/03/2017      14:21

Test	AC	Time
01 Room Air	0.000	14:23
02 *Subject Test	INT*	14:24
03 Room Air	0.000	14:25

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES, EDER

Remarks:

*Interferent*  
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005938  
Location = TOXL                    8164.14.00 09/16  
04/03/2017                            14:32

Test	AC	Time
01 Diagnostic	OK	14:34
02 Room Air	RFI*	14:34
03 Room Air	0.000	14:35

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005938  
Location = TOXL                    8164.14.00 09/16  
04/03/2017                            14:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:25
02 Std. Gas	0.082	14:26
03 Room Air	0.000	14:26
04 Std. Gas	0.082	14:27
05 Room Air	0.000	14:27
06 Std. Gas	0.082	14:27
07 Room Air	0.000	14:28

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005938  
Location = TOXL      8164.14.00 09/16  
04/03/2017      14:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:49
02 Std. Gas	0.082	14:49
03 Room Air	0.000	14:50
04 Std. Gas	0.081	14:50
05 Room Air	0.000	14:51
06 Std. Gas	0.082	14:51
07 Room Air	0.000	14:52

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005938  
Location = TOXL      8164.14.00 09/16  
04/03/2017      14:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:53
02 Std. Gas	0.083	14:53
03 Room Air	0.000	14:53
04 Std. Gas	0.082	14:54
05 Room Air	0.000	14:54
06 Std. Gas	0.082	14:55
07 Room Air	0.000	14:55

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000