



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number 80-005958	Instrument Location WCPD / McKenzie Co. Corrections
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) <u>moved to new building</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - ✓ A. Print Test (Level 1, Function P).
 - ✓ B. ACA Test (Level 1, Function C).
 - ✓ C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NA
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature  EISENCHENK	Date 04/07/2017
--	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by Deb Shanaver	Certified Date 10 Apr 17
--	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = WCPD 8164.13.00 06/09
04/07/2017 11:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 14
Standard Expiration? 09/05/2017
Oper No? 130409

Flow Cal. Date: 06/21/2013
Slope 716
Intercept -760436

IR Calibration Date: 04/07/2015
 3um 9um

0th Coef(*100): -17179 -17694
1st Coef(*100): 259328 130890
2nd Coef(*100): 3473 1654
H2O adj(mg/l*10k): 425 422

***** Printer Test End *****

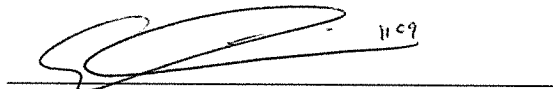

Operator Signature
ANDREW EISENSCHENK

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = WCPD 8164.13.00 06/09
04/07/2017 11:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:48
02 Std. Gas	0.079	11:49
03 Room Air	0.000	11:49
04 Std. Gas	0.079	11:50
05 Room Air	0.000	11:50
06 Std. Gas	0.080	11:50
07 Room Air	0.000	11:51

Lot No = 19615080A5
Cyl No = 14
Exp Date = 09/05/2017
County = 27 Oper No. = 130409



Operator Signature
ANDREW EISENSCHENK

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = WCPD 8164.13.00 06/09
04/07/2017 11:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:52
02 Room Air	0.000	11:52

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 14
Exp Date = 09/05/2017
County = 27 Oper No. = 130409

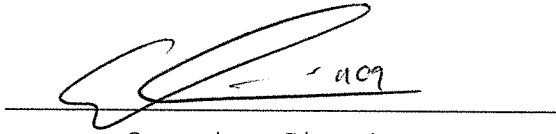

Operator Signature
ANDREW EISENSCHENK

Remarks: