



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-006505	Instrument Location Mohall, ND
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Moved to new location within office (bookings hall)	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature Roger Hutchinson	Date 04-10-17
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by Deb Shanaver	Certified Date 11 Apr 17
--	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
04/10/2017 12:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPNORSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPNORSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 31
Standard Expiration? 08/05/2018
Oper No? 131291

Flow Cal. Date: 08/20/2015
Slope 671
Intercept -642269

IR Calibration Date: 08/20/2015
3um 9um

0th Coef(*100): -29025 -17051
1st Coef(*100): 252705 133345
2nd Coef(*100): 3840 1546
H2O adj(mg/l*10k): 817 461

***** Printer Test End *****

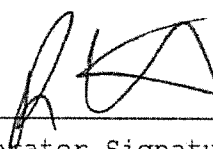

Operator Signature
ROGER HUTCHINSON

Remarks:

Instr 11


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
04/10/2017 12:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:48
02 Std. Gas	0.082	12:48
03 Room Air	0.000	12:49
04 Std. Gas	0.082	12:49
05 Room Air	0.000	12:49
06 Std. Gas	0.081	12:50
07 Room Air	0.000	12:50

Lot No = 17316080A2
Cyl No = 31
Exp Date = 08/05/2018
County = 38 Oper No. = 131291



Operator Signature
ROGER HUTCHINSON

Remarks:



Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
04/10/2017 13:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	13:03
02 Room Air	0.000	13:04


*Invalid Test
Inhibited - RFI

Lot No = 17316080A2
Cyl No = 31
Exp Date = 08/05/2018
County = 38 Oper No. = 131291

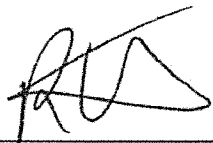

Operator Signature
ROGER HUTCHINSON

Remarks:

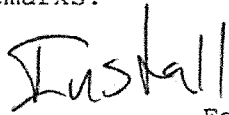

Form 106-I8000