

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004935

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3070 Lot No. 16060 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10 AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 8/5/2018  
Test 1 0.079 Test 4 0.080 Test 7 0.079  
Test 2 0.078 Test 5 0.079 Test 8 0.079  
Test 3 0.079 Test 6 0.079 Test 9 0.080  
Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 476 psi Reg 475 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED

SIMULATOR RETURN O-RING

4.5.2017  
Date

11 April  
Date

[Signature]  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004935  
Location = TOXL                    8164.14.00 09/16  
04/05/2017                            16:18

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            26  
Standard Expiration?       08/05/2018  
Oper No?                    666666

Flow Cal. Date:            07/12/2012  
Slope                      655  
Intercept                 -510289

IR Calibration Date:        05/20/2015

	3um	9um
0th Coef(*100):	-7871	-19684
1st Coef(*100):	264505	133108
2nd Coef(*100):	2865	1320
H2O adj(mg/l*10k):	200	298

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004935  
Location = TOXL                    8164.14.00 09/16  
04/05/2017                            16:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:22
02 Std. Sol.	0.019	16:23
03 Room Air	0.000	16:23
04 Std. Sol.	0.019	16:24
05 Room Air	0.000	16:24
06 Std. Sol.	0.018	16:25
07 Room Air	0.000	16:26

08 Sim Temp = 34.0°C

Simul Ser No = MP3003  
Std Sol No = 201610A  
County = 08                            Oper No. = 666666

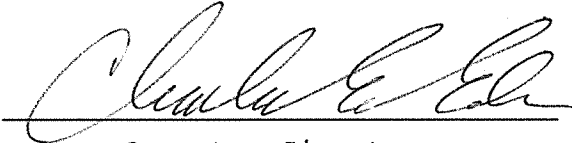

Operator Signature  
CHARLES EDER

Remarks:

*Low AC*  
*0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004935  
Location = TOXL      8164.14.00 09/16  
04/05/2017      16:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:28
02 Std. Sol.	0.295	16:29
03 Room Air	0.000	16:29
04 Std. Sol.	0.298	16:30
05 Room Air	0.000	16:30
06 Std. Sol.	0.299	16:31
07 Room Air	0.000	16:32

08 Sim Temp = 34.0°C

Simul Ser No = MP3070  
Std Sol No = 16060  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      HIGH AC  
                    0.300 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004935  
Location = TOXL      8164.14.00 09/16  
04/05/2017      16:36

Test	AC	Time
01 Room Air	0.000	16:37
02 *Subject Test	INT*	16:37
03 Room Air	0.000	16:38

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004935  
Location = TOXL                      8164.14.00 09/16  
04/05/2017                              16:39

Test	AC	Time
01 Diagnostic	OK	16:40
02 Room Air	0.000	16:40
03 *Subject Test	RFI*	16:40
04 Room Air	0.000	16:41

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                              Oper No. = 666666

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004935  
Location = TOXL                    8164.14.00 09/16  
04/05/2017                            16:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:42
02 Std. Gas	0.079	16:42
03 Room Air	0.000	16:43
04 Std. Gas	0.078	16:43
05 Room Air	0.000	16:43
06 Std. Gas	0.079	16:44
07 Room Air	0.000	16:44

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION  
CHECK 0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004935  
Location = TOXL      8164.14.00 09/16  
04/05/2017      16:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:45
02 Std. Gas	0.080	16:45
03 Room Air	0.000	16:46
04 Std. Gas	0.079	16:46
05 Room Air	0.000	16:47
06 Std. Gas	0.079	16:47
07 Room Air	0.000	16:48

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004935  
Location = TOXL                    8164.14.00 09/16  
04/05/2017                            16:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:48
02 Std. Gas	0.079	16:49
03 Room Air	0.000	16:49
04 Std. Gas	0.079	16:50
05 Room Air	0.000	16:50
06 Std. Gas	0.080	16:50
07 Room Air	0.000	16:51

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION  
CHECK 0.080 AC*