

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005939

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. IC54 Solution 0.10 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 8/5/2018
Test 1 0.082 Test 4 0.083 Test 7 0.083
Test 2 0.083 Test 5 0.081 Test 8 0.083
Test 3 0.082 Test 6 0.082 Test 9 0.082
Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 372 psi Reg 400 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED & TRAINED

EXTERNAL BATTERY (4.5.2017), REPLACED BREATH HOSE O-RING

4.6.2017
Date

[Signature]
Field Inspector's Signature

11 Apr 17
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 08:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:42
02 Std. Sol.	0.018	08:43
03 Room Air	0.000	08:44
04 Std. Sol.	0.019	08:44
05 Room Air	0.000	08:45
06 Std. Sol.	0.019	08:46
07 Room Air	0.000	08:46

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 08:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:48
02 Std. Sol.	0.403	08:49
03 Room Air	0.000	08:50
04 Std. Sol.	0.404	08:50
05 Room Air	0.000	08:51
06 Std. Sol.	0.402	08:52
07 Room Air	0.000	08:52

08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16102

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 08:54

Test	AC	Time
01 Room Air	0.000	08:55
02 *Subject Test	INT*	08:55
03 Room Air	0.000	08:56

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 09:07

Test	AC	Time
01 Diagnostic	OK	09:07
02 Room Air	0.000	09:08
03 *Subject Test	RFI*	09:08
04 Room Air	0.000	09:08

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

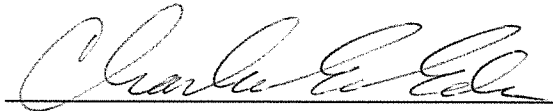

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 09:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:09
02 Std. Gas	0.082	09:10
03 Room Air	0.000	09:10
04 Std. Gas	0.083	09:10
05 Room Air	0.000	09:11
06 Std. Gas	0.082	09:11
07 Room Air	0.000	09:12

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

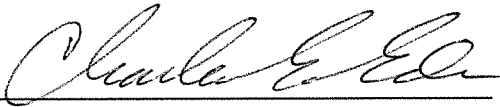
Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 09:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:13
02 Std. Gas	0.083	09:13
03 Room Air	0.000	09:14
04 Std. Gas	0.081	09:14
05 Room Air	0.000	09:15
06 Std. Gas	0.082	09:15
07 Room Air	0.000	09:15

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

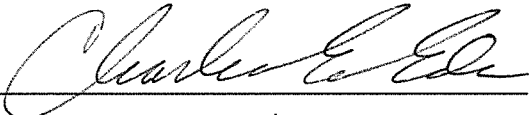
Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = TOXL 8164.14.00 09/16
04/06/2017 09:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:17
02 Std. Gas	0.083	09:17
03 Room Air	0.000	09:18
04 Std. Gas	0.083	09:18
05 Room Air	0.000	09:19
06 Std. Gas	0.082	09:19
07 Room Air	0.000	09:20

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

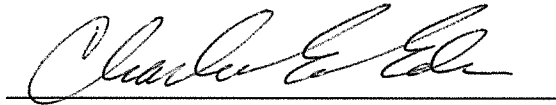

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000