

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003057

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR7111</u> Lot No. <u>16200</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10 AC + 0.05% Acetone</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>17316080A2</u>	Cyl No. <u>27</u>	Exp. Date <u>5 Aug 18</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 767 psi Reg 775 psi

K. Remarks/Maintenance record: Cleaned breath hose screen.
Changed breath hose o-ring and simulator return o-ring.
Annual inspection. Completed Flow Calibration.

18 Apr 17
Date

Roberta Wenger-Timm
Field Inspector's Signature

18 Apr 17
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

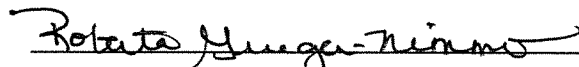
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 11/29/2010
Slope 653
Intercept -334316

IR Calibration Date: 04/15/2015
 3um 9um

0th Coef(*100): -23846 -15606
1st Coef(*100): 256102 135820
2nd Coef(*100): 2997 1528
H2O adj(mg/l*10k): 379 300

***** Printer Test End *****


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:32
02 Std. Sol.	0.019	14:33
03 Room Air	0.000	14:34
04 Std. Sol.	0.019	14:35
05 Room Air	0.000	14:35
06 Std. Sol.	0.019	14:36
07 Room Air	0.000	14:36

08 Sim Temp = 34.0°C

Simul Ser No = DR3846

Std Sol No = 16200

County = 08

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Low AC

0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

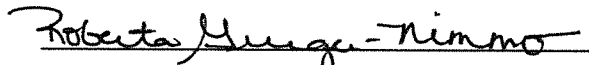
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:38
02 Std. Sol.	0.302	14:39
03 Room Air	0.000	14:39
04 Std. Sol.	0.303	14:40
05 Room Air	0.000	14:40
06 Std. Sol.	0.303	14:41
07 Room Air	0.000	14:42

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:49

Test	AC	Time
01 Room Air	0.000	14:50
02 *Subject Test	INT*	14:51
03 Room Air	0.000	14:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

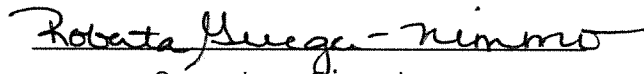
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:52

Test	AC	Time
01 Diagnostic	OK	14:53
02 Room Air	0.000	14:53
03 *Subject Test	RFI*	14:54
04 Room Air	0.000	14:55

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 14:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:56
02 Std. Gas	0.080	14:56
03 Room Air	0.000	14:57
04 Std. Gas	0.081	14:57
05 Room Air	0.000	14:58
06 Std. Gas	0.080	14:58
07 Room Air	0.000	14:58

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

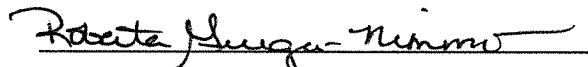
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 15:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:08
02 Std. Gas	0.080	15:08
03 Room Air	0.000	15:08
04 Std. Gas	0.081	15:09
05 Room Air	0.000	15:09
06 Std. Gas	0.080	15:10
07 Room Air	0.000	15:10

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

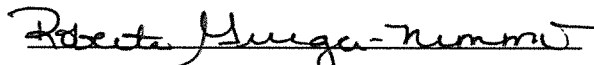
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
04/17/2017 15:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:13
02 Std. Gas	0.080	15:14
03 Room Air	0.000	15:14
04 Std. Gas	0.080	15:15
05 Room Air	0.000	15:15
06 Std. Gas	0.080	15:15
07 Room Air	0.000	15:16

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000