

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003066

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR1111</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10 AC + 0.05%</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>17316080A2</u>	Cyl No. <u>27</u>	Exp. Date <u>5 Aug 18</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 691 psi Reg 700 psi

K. Remarks/Maintenance record: Annual inspection; changed O-ring on simulator return.

18 Apr 17
Date

18 Apr 17
Date

Roberta Mueller-Nimmis
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 12:42

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 07/29/2010
Slope 806
Intercept -895033

IR Calibration Date: 07/29/2010

	3um	9um
0th Coef(*100):	-14075	-5808
1st Coef(*100):	272113	138204
2nd Coef(*100):	1637	986
H2O adj(mg/l*10k):	289	148

***** Printer Test End *****

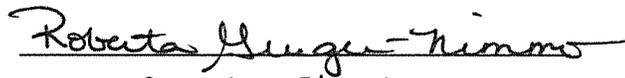

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 12:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Sol.	0.021	12:45
03 Room Air	0.000	12:46
04 Std. Sol.	0.021	12:46
05 Room Air	0.000	12:47
06 Std. Sol.	0.020	12:48
07 Room Air	0.000	12:48


08 Sim Temp = 34.0°C

Simul Ser No = DR3846

Std Sol No = 16200

County = 08

Oper No. = 888888

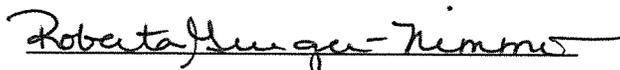

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 12:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:50
02 Std. Sol.	0.301	12:51
03 Room Air	0.000	12:51
04 Std. Sol.	0.301	12:52
05 Room Air	0.000	12:53
06 Std. Sol.	0.302	12:53
07 Room Air	0.000	12:54

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888

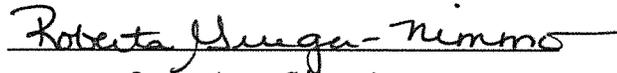
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 13:00

Test	AC	Time
01 Diagnostic	OK	13:01
02 Room Air	0.000	13:01
03 Subject Test 1	0.000	13:03
04 Room Air	RFI*	13:04
05 Room Air	0.000	13:04

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

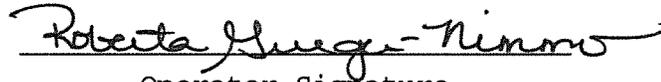
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *RFI Check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 13:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:05
02 Std. Gas	0.080	13:05
03 Room Air	0.000	13:06
04 Std. Gas	0.080	13:06
05 Room Air	0.000	13:07
06 Std. Gas	0.080	13:07
07 Room Air	0.000	13:08

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 13:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:09
02 Std. Gas	0.080	13:09
03 Room Air	0.000	13:10
04 Std. Gas	0.079	13:10
05 Room Air	0.000	13:11
06 Std. Gas	0.080	13:11
07 Room Air	0.000	13:11

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

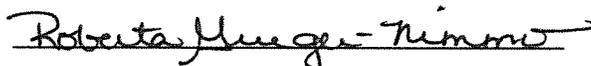
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = TOXL 8164.14.00 09/16
04/18/2017 13:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:13
02 Std. Gas	0.080	13:13
03 Room Air	0.000	13:14
04 Std. Gas	0.079	13:14
05 Room Air	0.000	13:14
06 Std. Gas	0.080	13:15
07 Room Air	0.000	13:15


Lot No = 17316080A2

Cyl No = 27

Exp Date = 08/05/2018

County = 08

Oper No. = 888888

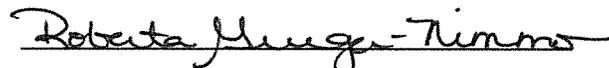

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000