

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004188

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.083</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.083</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 1003 psi Reg 1000 psi

K. Remarks/Maintenance record: REPLACED SIMULATOR RETURN ORING,

ANNUAL INSPECTION

4-18-17
Date

18 Apr 17
Date

Charles E. Egan
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:25

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 07/05/2011
Slope 668
Intercept -597204

IR Calibration Date: 04/05/2012
 3um 9um

0th Coef(*100): -15090 -16406
1st Coef(*100): 263383 126966
2nd Coef(*100): 6533 2371
H2O adj(mg/l*10k): 801 809

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:29
02 Std. Sol.	0.019	12:29
03 Room Air	0.000	12:30
04 Std. Sol.	0.019	12:31
05 Room Air	0.000	12:31
06 Std. Sol.	0.020	12:32
07 Room Air	0.000	12:32

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:39
02 Std. Sol.	0.408	12:40
03 Room Air	0.000	12:40
04 Std. Sol.	0.408	12:41
05 Room Air	0.000	12:42
06 Std. Sol.	0.408	12:42
07 Room Air	0.000	12:43

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666

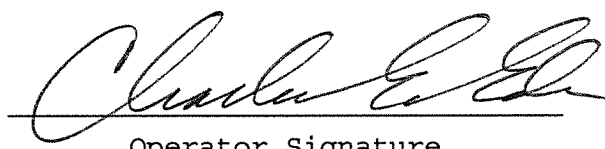

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.400 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:47

Test	AC	Time
01 Room Air	0.000	12:48
02 *Subject Test	INT*	12:48
03 Room Air	0.000	12:49

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:49

Test	AC	Time
01 Diagnostic	OK	12:50
02 Room Air	0.000	12:50
03 *Subject Test	RFI*	12:51
04 Room Air	0.000	12:51

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:52
02 Std. Gas	0.082	12:52
03 Room Air	0.000	12:53
04 Std. Gas	0.082	12:53
05 Room Air	0.000	12:54
06 Std. Gas	0.082	12:54
07 Room Air	0.000	12:55

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

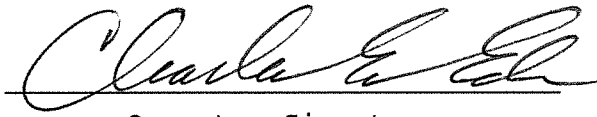

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 12:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:55
02 Std. Gas	0.082	12:56
03 Room Air	0.000	12:56
04 Std. Gas	0.082	12:57
05 Room Air	0.000	12:57
06 Std. Gas	0.083	12:58
07 Room Air	0.000	12:58

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

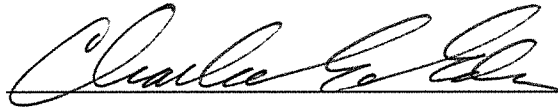

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.14.00 09/16
04/18/2017 13:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:04
02 Std. Gas	0.082	13:05
03 Room Air	0.000	13:05
04 Std. Gas	0.083	13:06
05 Room Air	0.000	13:06
06 Std. Gas	0.083	13:06
07 Room Air	0.000	13:07

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC