

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003061

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR7111</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10Ac + 0.05%
Acetone</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18

Test 1 0.079 Test 4 0.080 Test 7 0.080

Test 2 0.080 Test 5 0.080 Test 8 0.079

Test 3 0.080 Test 6 0.080 Test 9 0.080

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 753 psi Reg 750 psi

K. Remarks/Maintenance record: Annual inspection; replaced O-ring
on simulator return.

18 Apr 17
Date

18 Apr 17
Date

Roberta Hunger-Tummo
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:16

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 03/04/2011
Slope 700
Intercept -623726

IR Calibration Date: 04/10/2015
 3um 9um

0th Coef(*100): -25722 -17879
1st Coef(*100): 266163 136703
2nd Coef(*100): 2302 1151
H2O adj(mg/l*10k): 412 294

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Sol.	0.019	11:19
03 Room Air	0.000	11:19
04 Std. Sol.	0.018	11:20
05 Room Air	0.000	11:21
06 Std. Sol.	0.019	11:21
07 Room Air	0.000	11:22

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *LOW AC*
0.020 AC

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Sol.	0.299	11:24
03 Room Air	0.000	11:24
04 Std. Sol.	0.299	11:25
05 Room Air	0.000	11:26
06 Std. Sol.	0.299	11:26
07 Room Air	0.000	11:27


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 16060

County = 08

Oper No. = 888888

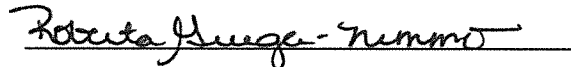

Operator Signature

ROBERTA GRIEGER-NIMMO

Remarks:

High AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:28

Test	AC	Time
01 Room Air	0.000	11:29
02 *Subject Test	INT*	11:29
03 Room Air	0.000	11:30

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

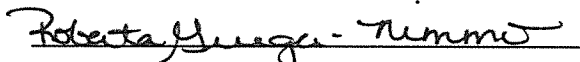
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:30

Test	AC	Time
01 Diagnostic	OK	11:31
02 Room Air	RFI*	11:31
03 Room Air	0.000	11:32

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:32
02 Std. Gas	0.079	11:33
03 Room Air	0.000	11:33
04 Std. Gas	0.080	11:34
05 Room Air	0.000	11:34
06 Std. Gas	0.080	11:35
07 Room Air	0.000	11:35

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

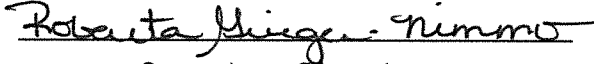
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Gas	0.080	11:36
03 Room Air	0.000	11:37
04 Std. Gas	0.080	11:37
05 Room Air	0.000	11:38
06 Std. Gas	0.080	11:38
07 Room Air	0.000	11:39

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC
Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2017 11:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:39
02 Std. Gas	0.080	11:40
03 Room Air	0.000	11:40
04 Std. Gas	0.079	11:41
05 Room Air	0.000	11:41
06 Std. Gas	0.080	11:42
07 Room Air	0.000	11:42

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

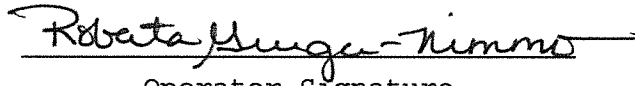

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC