

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004196

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DRS135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 862 psi Reg 825 psi

K. Remarks/Maintenance record: REPLACED BREATH HOSE AND SIMULATOR RETURN

0-RINGS, ANNUAL INSPECTIONS

4-19-17
Date

19 Apr 17
Date

Charles E. Edgerly
Field Inspector's Signature

Debs Shenaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 06/04/2009
Slope 727
Intercept -600779

IR Calibration Date: 06/02/2015
 3um 9um

0th Coef(*100): -19524 -13707
1st Coef(*100): 263393 129371
2nd Coef(*100): 2924 1442
H2O adj(mg/l*10k): 392 311

***** Printer Test End *****

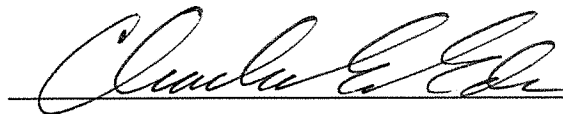

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:35
02 Std. Sol.	0.019	10:36
03 Room Air	0.000	10:37
04 Std. Sol.	0.020	10:37
05 Room Air	0.000	10:38
06 Std. Sol.	0.019	10:39
07 Room Air	0.000	10:39

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:41
02 Std. Sol.	0.392	10:42
03 Room Air	0.000	10:43
04 Std. Sol.	0.395	10:43
05 Room Air	0.000	10:44
06 Std. Sol.	0.396	10:45
07 Room Air	0.000	10:45

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666

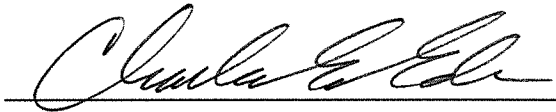

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.400 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:47

Test	AC	Time
01 Room Air	0.000	10:48
02 *Subject Test	INT*	10:49
03 Room Air	0.000	10:49

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

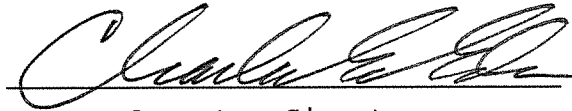

Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:50

Test	AC	Time
01 Diagnostic	OK	10:51
02 Room Air	0.000	10:51
03 *Subject Test	RFI*	10:51
04 Room Air	0.000	10:52

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Gas	0.080	10:53
03 Room Air	0.000	10:54
04 Std. Gas	0.080	10:54
05 Room Air	0.000	10:55
06 Std. Gas	0.080	10:55
07 Room Air	0.000	10:56

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 10:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Gas	0.080	10:57
03 Room Air	0.000	10:57
04 Std. Gas	0.080	10:58
05 Room Air	0.000	10:58
06 Std. Gas	0.080	10:59
07 Room Air	0.000	10:59

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080AA

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
04/19/2017 11:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:02
02 Std. Gas	0.080	11:03
03 Room Air	0.000	11:03
04 Std. Gas	0.080	11:03
05 Room Air	0.000	11:04
06 Std. Gas	0.081	11:04
07 Room Air	0.000	11:05

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC