

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004950

Location: TOXL

- | | Check When Done |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR7111</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10AC+0.05%</u>
<u>Acetone</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 644 psi Reg 650 psi

K. Remarks/Maintenance record: Annual inspection; replaced o-rings on
breath hose and simulator return.

19 Apr 17
Date

19 Apr 17
Date

Roberta Guegu-Timmis
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 09:54

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 05/05/2016
 3um 9um

0th Coef (*100): -13717 -19495
1st Coef (*100): 271916 140513
2nd Coef (*100): 3504 1503
H2O adj (mg/l*10k): 370 426

***** Printer Test End *****

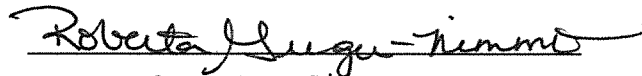

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:16
02 Std. Sol.	0.020	10:16
03 Room Air	0.000	10:17
04 Std. Sol.	0.020	10:18
05 Room Air	0.000	10:18
06 Std. Sol.	0.020	10:19
07 Room Air	0.000	10:19

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 888888

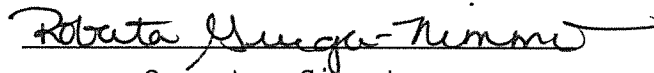
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:22
02 Std. Sol.	0.306	10:23
03 Room Air	0.000	10:23
04 Std. Sol.	0.307	10:24
05 Room Air	0.000	10:25
06 Std. Sol.	0.305	10:25
07 Room Air	0.000	10:26

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888

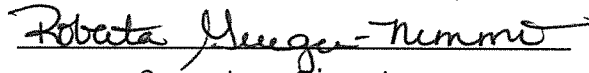

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:27

Test	AC	Time
01 Room Air	0.000	10:28
02 *Subject Test	INT*	10:29
03 Room Air	0.000	10:29

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

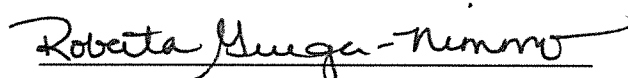

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:30

Test	AC	Time
01 Diagnostic	OK	10:30
02 Room Air	0.000	10:31
03 Subject Test 1	0.000	10:31
04 Room Air	0.000	10:33
05 Reference	RFI*	10:33
06 Room Air	0.000	10:34

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

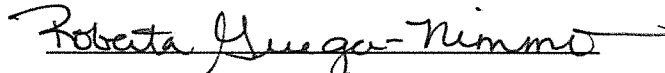

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Gas	0.080	10:36
03 Room Air	0.000	10:37
04 Std. Gas	0.081	10:37
05 Room Air	0.000	10:37
06 Std. Gas	0.081	10:38
07 Room Air	0.000	10:38

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

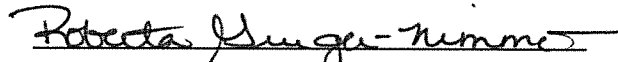
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Gas	0.081	10:39
03 Room Air	0.000	10:40
04 Std. Gas	0.081	10:40
05 Room Air	0.000	10:41
06 Std. Gas	0.081	10:41
07 Room Air	0.000	10:42

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

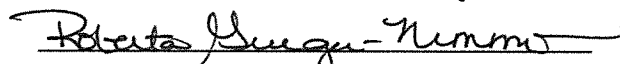
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/19/2017 10:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:43
02 Std. Gas	0.081	10:43
03 Room Air	0.000	10:44
04 Std. Gas	0.080	10:44
05 Room Air	0.000	10:44
06 Std. Gas	0.081	10:45
07 Room Air	0.000	10:45

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

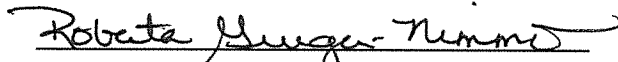
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
 0.080 AC
Form 106-I8000