

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003069

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR7111</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10 AC + 0.05%</u>
<u>Acetone</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>17316080A2</u>	Cyl No. <u>27</u>	Exp. Date <u>5 Aug 18</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.081</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 543 psi Reg 550 psi

K. Remarks/Maintenance record: Annual inspection, replaced o-ring on
breath hose.

19 Apr 17
Date

19 Apr '17
Date

Roberta Muegge-Thimm
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:08

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

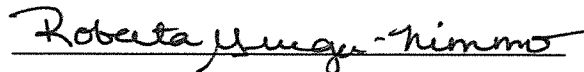
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 09/17/2013
Slope 678
Intercept -734495

IR Calibration Date: 04/08/2015

	3um	9um
0th Coef(*100):	-13291	-10923
1st Coef(*100):	270590	132326
2nd Coef(*100):	4128	1532
H2O adj(mg/l*10k):	348	226

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

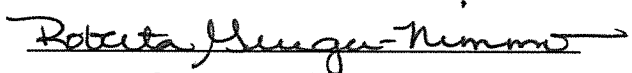
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Sol.	0.020	14:11
03 Room Air	0.000	14:12
04 Std. Sol.	0.020	14:12
05 Room Air	0.000	14:13
06 Std. Sol.	0.020	14:14
07 Room Air	0.000	14:14

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:16
02 Std. Sol.	0.305	14:17
03 Room Air	0.000	14:17
04 Std. Sol.	0.307	14:18
05 Room Air	0.000	14:19
06 Std. Sol.	0.307	14:19
07 Room Air	0.000	14:20

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000

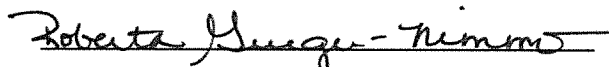
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:21

Test	AC	Time
01 Room Air	0.000	14:22
02 *Subject Test	INT*	14:22
03 Room Air	0.000	14:23

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

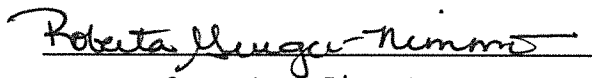
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:23

Test	AC	Time
01 Diagnostic	OK	14:24
02 Room Air	0.000	14:24
03 *Subject Test	RFI*	14:25
04 Room Air	0.000	14:25

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

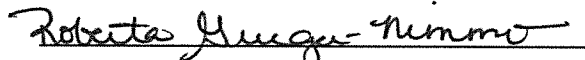
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Gas	0.080	14:26
03 Room Air	0.000	14:27
04 Std. Gas	0.080	14:27
05 Room Air	0.000	14:28
06 Std. Gas	0.080	14:28
07 Room Air	0.000	14:29

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.080	14:30
03 Room Air	0.000	14:30
04 Std. Gas	0.080	14:31
05 Room Air	0.000	14:31
06 Std. Gas	0.079	14:31
07 Room Air	0.000	14:32

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.14.00 09/16
04/19/2017 14:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:33
02 Std. Gas	0.080	14:33
03 Room Air	0.000	14:34
04 Std. Gas	0.080	14:34
05 Room Air	0.000	14:34
06 Std. Gas	0.081	14:35
07 Room Air	0.000	14:35

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC