

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004200

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3070 Lot No. 16060 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018  
Test 1 0.083 Test 4 0.081 Test 7 0.082  
Test 2 0.082 Test 5 0.082 Test 8 0.082  
Test 3 0.082 Test 6 0.082 Test 9 0.082  
Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 638 psi Reg 625 psi

K. Remarks/Maintenance record: REPLACED SIMULATOR RETURN O-RING,

ANNUAL INSPECTION

4-21-17  
Date

04 May 17  
Date

Charles E. Lee  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-004200  
Location = TOXL 8164.14.00 09/16  
04/21/2017 16:08

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 17316080A2  
Standard Cyl #? 28  
Standard Expiration? 08/05/2018  
Oper No? 666666

Flow Cal. Date: 12/10/2012  
Slope 637  
Intercept -382487

IR Calibration Date: 12/10/2012  
3um 9um

	3um	9um
0th Coef(*100):	-14942	-22738
1st Coef(*100):	256196	136921
2nd Coef(*100):	3535	1396
H2O adj(mg/l*10k):	492	515

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
04/21/2017      16:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:10
02 Std. Sol.	0.018	16:11
03 Room Air	0.000	16:11
04 Std. Sol.	0.018	16:12
05 Room Air	0.000	16:13
06 Std. Sol.	0.018	16:13
07 Room Air	0.000	16:14

08 Sim Temp = 34.0°C

Simul Ser No = MP3003  
Std Sol No = 201610A  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
04/21/2017      16:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:20
02 Std. Sol.	0.302	16:21
03 Room Air	0.000	16:22
04 Std. Sol.	0.303	16:22
05 Room Air	0.000	16:23
06 Std. Sol.	0.302	16:24
07 Room Air	0.000	16:24

08 Sim Temp = 34.0°C

Simul Ser No = MP3070  
Std Sol No = 16060  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
04/21/2017      16:26

Test	AC	Time
01 Room Air	0.000	16:26
02 *Subject Test	INT*	16:27
03 Room Air	0.000	16:27

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666

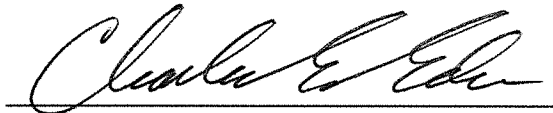

Operator Signature  
CHARLES EDER

Remarks:

*R* Interferent  
*cec* 0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004200  
Location = TOXL                8164.14.00 09/16  
04/21/2017                        16:28

Test	AC	Time
01 Diagnostic	OK	16:29
02 Room Air	0.000	16:29
03 *Subject Test	RFI*	16:29
04 Room Air	0.000	16:30

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004200  
Location = TOXL                    8164.14.00 09/16  
04/21/2017                            16:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:31
02 Std. Gas	0.083	16:31
03 Room Air	0.000	16:31
04 Std. Gas	0.082	16:32
05 Room Air	0.000	16:32
06 Std. Gas	0.082	16:33
07 Room Air	0.000	16:33

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08                            Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
04/21/2017      16:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:34
02 Std. Gas	0.081	16:34
03 Room Air	0.000	16:35
04 Std. Gas	0.082	16:35
05 Room Air	0.000	16:36
06 Std. Gas	0.082	16:36
07 Room Air	0.000	16:37

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
04/21/2017      16:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:37
02 Std. Gas	0.082	16:38
03 Room Air	0.000	16:38
04 Std. Gas	0.082	16:39
05 Room Air	0.000	16:39
06 Std. Gas	0.082	16:39
07 Room Air	0.000	16:40

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

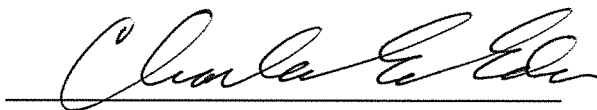

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000