

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004205

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR7111</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>G10329</u> Lot No. <u>1CS4</u> Solution <u>0.10AC + 0.05% Acetone</u> | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18
Test 1 0.078 Test 4 0.078 Test 7 0.078
Test 2 0.078 Test 5 0.078 Test 8 0.078
Test 3 0.079 Test 6 0.079 Test 9 0.078
Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 515 psi Reg 500 psi

K. Remarks/Maintenance record: Annual inspection; replaced and
external battery. Replaced O-ring on simulator return and
breath hose inlet.
trained
20 Apr 17 Date Roberta Young-Nimmis Field Inspector's Signature
04 May 17 Date Deb Shanaver Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 07:56

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 06/04/2009
Slope 704
Intercept -610062

IR Calibration Date: 04/17/2015

	3um	9um
0th Coef(*100):	-26411	-27902
1st Coef(*100):	266995	129434
2nd Coef(*100):	2919	1480
H2O adj(mg/l*10k):	363	481

***** Printer Test End *****

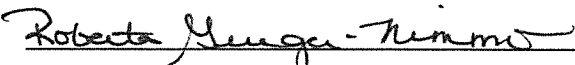
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 07:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:58
02 Std. Sol.	0.020	07:59
03 Room Air	0.000	07:59
04 Std. Sol.	0.020	08:00
05 Room Air	0.000	08:01
06 Std. Sol.	0.020	08:01
07 Room Air	0.000	08:02


08 Sim Temp = 34.0°C

Simul Ser No = DR3846

Std Sol No = 16200

County = 08

Oper No. = 888888

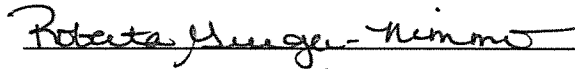

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 08:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:05
02 Std. Sol.	0.300	08:05
03 Room Air	0.000	08:06
04 Std. Sol.	0.301	08:07
05 Room Air	0.000	08:07
06 Std. Sol.	0.301	08:08
07 Room Air	0.000	08:08

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888

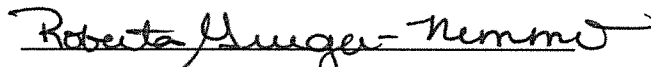
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 09:50

Test	AC	Time
01 Room Air	0.000	09:50
02 *Subject Test	INT*	09:51
03 Room Air	0.000	09:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 09:55

Test	AC	Time
01 Diagnostic	OK	09:56
02 Room Air	0.000	09:56
03 Subject Test 1	0.000	09:57
04 Room Air	RFI*	09:58
05 Room Air	0.000	09:59

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

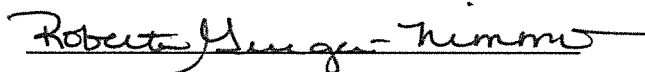

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 10:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:04
02 Std. Gas	0.078	10:05
03 Room Air	0.000	10:05
04 Std. Gas	0.078	10:05
05 Room Air	0.000	10:06
06 Std. Gas	0.079	10:06
07 Room Air	0.000	10:07

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

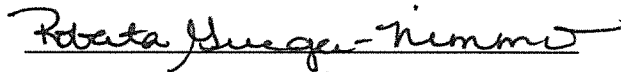

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 10:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Gas	0.078	10:09
03 Room Air	0.000	10:09
04 Std. Gas	0.078	10:09
05 Room Air	0.000	10:10
06 Std. Gas	0.079	10:10
07 Room Air	0.000	10:11

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/20/2017 10:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:12
02 Std. Gas	0.078	10:12
03 Room Air	0.000	10:12
04 Std. Gas	0.078	10:13
05 Room Air	0.000	10:13
06 Std. Gas	0.078	10:14
07 Room Air	0.000	10:14

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC