

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005952

Location: Burleigh County SD

- Check When Done
- A. Is the breath tube heated? X
 - B. Is the diagnostic check complete? X
 - C. Is the time, date, and year correct (re-set if necessary)? X
 - D. Print test. (Attach test record.) X
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.) 0.020
Sim SN DR5113 Lot No. 201507F Rep AC 28 July 17 Resu X
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3378 Lot No. 16060 Rep AC 0.300 X
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR3803 Lot No. 1CS 4 Solution 0.10 AC + 0.05% X
Acetone
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) X
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 09216080A3 cyl No. 31 Exp. Date 5 May 18

Test 1 0.082 Test 4 0.082 Test 7 0.082

Test 2 0.082 Test 5 0.082 Test 8 0.082

Test 3 0.081 Test 6 0.081 Test 9 0.081

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 761 psi Reg 750 psi

K. Remarks/Maintenance record: Annual inspection; changed
O-ring on breath hose.

5 May 17
Date

Roberta Mungas-Mimmo
Field Inspector's Signature

05 May 17
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:11

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

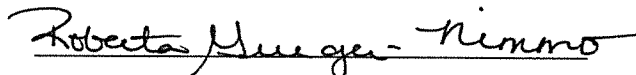
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 31
Standard Expiration? 05/05/2018
Oper No? 888888

Flow Cal. Date: 06/21/2013
Slope 734
Intercept -865305

IR Calibration Date: 06/21/2013
 3um 9um

0th Coef(*100):	-21281	-23057
1st Coef(*100):	292132	137092
2nd Coef(*100):	2334	1231
H2O adj(mg/l*10k):	497	482

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

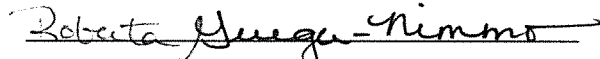
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:33
02 Std. Sol.	0.020	11:34
03 Room Air	0.000	11:34
04 Std. Sol.	0.019	11:35
05 Room Air	0.000	11:35
06 Std. Sol.	0.020	11:36
07 Room Air	0.000	11:37

08 Sim Temp = 34.0°C

Simul Ser No = DR5113
Std Sol No = 201507F
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:39
02 Std. Sol.	0.305	11:39
03 Room Air	0.000	11:40
04 Std. Sol.	0.306	11:41
05 Room Air	0.000	11:41
06 Std. Sol.	0.307	11:42
07 Room Air	0.000	11:42

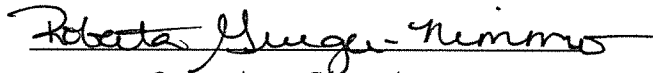
08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 08

Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000

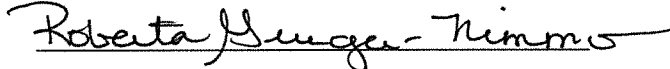
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:26

Test	AC	Time
01 Room Air	0.000	11:27
02 *Subject Test	INT*	11:28
03 Room Air	0.000	11:28

*Invalid Test
Interferent Detected

Sub Name = INTERFERENT, NA NA
Sub DOB = 11/11/2000
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 09216080A3
Cyl No = 31
Expiration Date = 05/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

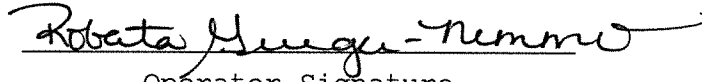
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:11

Test	AC	Time
01 Diagnostic	OK	11:13
02 Room Air	RFI*	11:13
03 Room Air	0.000	11:13

*Invalid Test
 Inhibited - RFI

Sub Name = RFI, NA NA
Sub DOB = 11/11/2000
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 09216080A3
Cyl No = 31
Expiration Date = 05/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

RFI Check

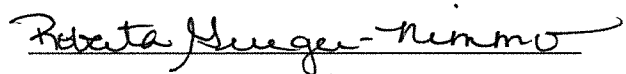
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:14
02 Std. Gas	0.082	11:15
03 Room Air	0.000	11:15
04 Std. Gas	0.082	11:16
05 Room Air	0.000	11:16
06 Std. Gas	0.081	11:17
07 Room Air	0.000	11:17

Lot No = 09216080A3
Cyl No = 31
Exp Date = 05/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

Calibration Check
0.080 AC
Form 106-I8000

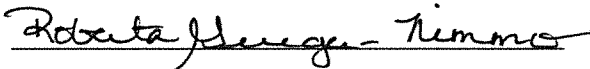
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Gas	0.082	11:18
03 Room Air	0.000	11:19
04 Std. Gas	0.082	11:19
05 Room Air	0.000	11:20
06 Std. Gas	0.081	11:20
07 Room Air	0.000	11:21

Lot No = 09216080A3
Cyl No = 31
Exp Date = 05/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC
Form 106-I8000

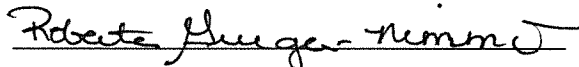
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
05/05/2017 11:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Gas	0.082	11:23
03 Room Air	0.000	11:24
04 Std. Gas	0.082	11:24
05 Room Air	0.000	11:25
06 Std. Gas	0.081	11:25
07 Room Air	0.000	11:26

Lot No = 09216080A3
Cyl No = 31
Exp Date = 05/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000