

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-002666

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080AC

Lot No. 17316080A2 Cyl No. 028 Exp. Date 8/5/2018

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.083</u>
Test 2 <u>0.083</u>	Test 5 <u>0.082</u>	Test 8 <u>0.083</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.082</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 584 psi Reg 575 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/5/2017
Date

8 May 17
Date

Charles E. Ed
Field Inspector's Signature

Roberta Huggins-Nimmo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 12/31/2012
Slope 657
Intercept -418473

IR Calibration Date: 05/27/2016

	3um	9um
0th Coef(*100):	-18254	-11303
1st Coef(*100):	275128	137601
2nd Coef(*100):	2576	1178
H2O adj(mg/l*10k):	442	273

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Sol.	0.019	16:28
03 Room Air	0.000	16:29
04 Std. Sol.	0.019	16:29
05 Room Air	0.000	16:30
06 Std. Sol.	0.019	16:31
07 Room Air	0.000	16:31


08 Sim Temp = 34.0°C

Simul Ser No = MP3003

Std Sol No = 201610A

County = 08

Oper No. = 666666

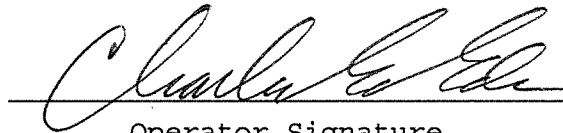

Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Sol.	0.399	16:34
03 Room Air	0.000	16:34
04 Std. Sol.	0.398	16:35
05 Room Air	0.000	16:35
06 Std. Sol.	0.398	16:36
07 Room Air	0.000	16:37

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.400 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:38

Test	AC	Time
01 Room Air	0.000	16:39
02 *Subject Test	INT*	16:39
03 Room Air	0.000	16:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:



0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:43
02 Std. Gas	0.082	16:43
03 Room Air	0.000	16:44
04 Std. Gas	0.083	16:44
05 Room Air	0.000	16:45
06 Std. Gas	0.082	16:45
07 Room Air	0.000	16:46

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:47
02 Std. Gas	0.082	16:47
03 Room Air	0.000	16:47
04 Std. Gas	0.082	16:48
05 Room Air	0.000	16:48
06 Std. Gas	0.083	16:49
07 Room Air	0.000	16:49

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = TOXL 8164.14.00 09/16
05/05/2017 16:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:50
02 Std. Gas	0.083	16:50
03 Room Air	0.000	16:51
04 Std. Gas	0.083	16:51
05 Room Air	0.000	16:52
06 Std. Gas	0.082	16:52
07 Room Air	0.000	16:53

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC